

User Manual of bright Chair 2



I083-01MU-02 rev.0

Revision History

Rev. No.	Rev. Date	Description
0	2023.08.11	First Edition

Contents

1. Precautions for safety	6
■ Precautions before product installation.....	6
■ Precautions before use.....	6
■ Precautions for Product Operation.....	7
■ Precautions when using the product	8
■ Electrical Precautions.....	9
■ Warning	9
■ Manufacturer's Obligation Policy	9
■ Safety Instructions	10
2. Name of each part	12
2.1 bright Chair 2 (ICT-CH02S)	12
3. How to operate each part	13
3.1 How to use the power system	13
3.2 Dr. table.....	14
3.2.1 How to use Dr. Table	14
A. How to use the control panel	15
B. LCD Description.....	18
3.2.2 How to adjust the water quantity of the Handpiece	20
3.2.3 How to use the High Handpiece.....	21
3.2.4 How to use the Low Handpiece	22
3.2.5 How to use Electric Motor (Option ICT-BE1).....	23
3.2.6 Usage environment and conditions	26
3.2.7 Periodic inspection	26
3.2.8 How to connect and disconnect the motor and Handpiece	26
3.2.9 Cleaning and disinfection methods	27
3.2.10 Sterilization method	29
3.2.11 Maintenance	29
3.2.12 How to use the Scaler(bright Scaler_ICT-BS1)	30
A. Main components	30
3.3 Auxiliary Tables	34
3.3.1 Names of Parts.....	34

3.3.2 How to use the auxiliary table	35
3.3.3 Suction Syringe (Big/Small) How to use	36
3.3.4 3-Way Syringe How to use	37
3.3.5 How to remove Syringe Tip	38
3.3.6 Cleaning and disinfection methods	39
3.3.7 Storage method	40
3.3.8 How to use single-person treatment suction	40
3.5 Dental light.....	41
3.5.1 Appearance	42
3.6 Watering Device	43
3.7 Headrest.....	46
3.8 Foot controller	47
3.8.1 Functional Foot Controller	47
3.8.2 Disc type foot controller.....	48
3.9 Emergency operation guide	49
3.9.1 Guide for Operation	49
3.10 Cuspidor table rotation detection guidance	50
4. Maintenance	51
4.1 Replacement of Oil Can of Dr. table	51
4.2 How to clean the unit	52
4.3 Cleaning the spittoon & spittoon filter.....	53
4.4 Cleaning the suction filter	53
4.5 Sanitation(Optional)	54
5. Maintenance after use.....	61
6. Problem solving.....	63
7. Product Specifications and Specifications	65
7.1 Dental unit chair	65
7.2 Electric Motor	66
7.3 Dental light.....	66
7.4 bright scaler	67

8. labeling	68
8.1 Labels	68
9. Components	71
9.1 Basic Components.....	71
9.1.1 Tissue & Cup Holder.....	72
9.2 Optional Components.....	73
9.2.1 Monitor Arm.....	73
9.3 Reference	74
10. Appendix	75
10.1 Precautions for disposal of the system	75
10.2 Electronic Appliance Compatible Information	76
10.3 Installation and precautions.....	79
10.4 Manufacturer.....	80
10.5 Warranty and Service.....	80

Manual guide

Thank you for purchasing Dentium bright Chair 2, the latest dental unit and chair (ICT-CH02S).

This manual describes in detail how to use, how to handle, or what is related to general use.

Please read this manual thoroughly in advance to use the equipment smoothly.

Please keep the manual around the equipment for reference at any time.

The manual is subject to change without notice due to product quality upgrades and specification changes.

Before using the product, be sure to read "Safety Precautions" and use the equipment correctly.

Manual Version: 0

Manual Issue Date: 2023.08.11

Tel: +82-70-7098-8962, Fax +82-31-8019-9106

Website: www.dentium.co.kr

Address: (16229) 76, Changnyong-daero 256beon-gil, Yeongtong-gu, Suwon-si, Gyeonggi-do,

Republic of Korea

Product description

- This product is a dental Unit and chair equipped with the necessary equipment to be used for various dental treatments.

Intended Purpose

- Dentium bright Chair 2 is a dental unit chair. The system is intended for use in dental care medical operations. The system is to be used by authorised professionals within the scope of his/her education, training and experience.

The system provides the dental practitioner a motorised patient chair, dental instruments and suction system for removal of bodily fluids.

This product is a device that moves the patient to the medical treatment position and drives the medical treatment device to enable the dental treatment.

1. Precautions for safety

Precautions before product installation

- Install in a place where power cords such as the equipment's power cord are not damaged and avoid doors or passages.
- Do not use old outlets and damaged power cords or plugs.
- Do not drop the device or subject it to impact.
- Avoid installation in damp or dusty places as much as possible.
- Do not install the device in an unstable location subject to slope, vibration, impact, etc.
- Please install on a flat floor.
- Please use a grounded outlet.

Precautions before use

- Ensure that the medical device is operated by a qualified person
- Be sure to read the instruction manual before use.
- Disinfect all accessories that require disinfection before use.
- When using the equipment, be sure to check the operating conditions, status check, and the distance from the patient.

- Do not modify the equipment without the manufacturer's permission. Damage to the device due to user disassembly is not covered under warranty even within the warranty period.
- Do not place similar medical electrical devices near the equipment. It may degrade the performance of your device.
- Keep away from other sources of electromagnetic energy to avoid electromagnetic interference
- If you use a product other than the recommended product, it may cause performance degradation or malfunction.

Precautions for Product Operation

- Be sure to keep the handpiece in the dedicated cradle after the procedure.
- Be careful not to touch with your hands or other objects while the surgical instrument is rotating.
- Do not assemble or disassemble the handpiece while the device is in operation.
- Connect the handpiece after the micromotor and related devices have completely stopped working.
- Do not apply excessive load to the equipment, as excessive force or excessive force may cause malfunction or deterioration of the equipment.
- If noise or heat generation occurs during use, stop using it immediately and contact the nearest Dentium branch or manufacturer to check before use.
- If you use a product other than the recommended product, it may cause performance degradation or malfunction.
- Do not drop the handpiece or subject it to strong external impact.
- This product is not a medical device to provide heat to the patient.
- The product is applicable to healthy skin of adults.
- People with poor blood flow (vascular disorders, blood circulation disorders, diabetes, high blood pressure) or weak skin sensitivity are prone to low-temperature burns.
- For inspection and repair of the device, contact the point of purchase or the manufacturer.
- Do not put objects in the movable parts or range of movement of the body, such as the joint of the backrest under the seat or put your fingers or feet. If it gets caught during operation, it may cause injury or damage or malfunction of the device.
- Do not apply excessive load to the table arm or light arm, or place heavy objects on, lean on, or sit on the end of the seat, backrest, headrest, or table, auxiliary tray, or mouse tray. The fall or breakage of the device may cause injury to the user or malfunction of the device.
- Observe safety precautions and ensure proper use of the device while operating the device.
- Do not install the device in an unsafe place subject to inclination, vibration, or impact.
- Do not operate the device except for qualified personnel.

- Avoid any movements or postures that endanger the patient.
- Be careful not to get your fingers or body parts caught in the moving parts of the main body.
- Do not allow more than two people to sit on the seat. Climbing up with a child can be dangerous.
- Do not allow people not related to medical care to approach the device.
- If there is a risk such as a collision while operating the seat of the body, press one of the chair operation keys on the table control panel on Dr. Table's side or the auxiliary table's side to perform an emergency stop operation.
- For inspection and repair of the device, contact the point of purchase or the manufacturer.

Precautions when using the product

- The operation of the treatment table may cause injury or harm to the patient and the operator.
- Be careful of both the patient and the operator when the treatment table is operating.
- Actions such as jumping on the treatment table can be dangerous.
- Do not apply more than 20N (3kgf) load to Dr. table.
- Do not apply more than 10N(kgf) load to the auxiliary table.
- If an inappropriate medical system device is connected to the medical device, it may cause an electric shock. When connecting IT devices to medical equipment, comply with the provisions of IEC60601-1 and IEC60601-1-2.
- Do not stick stickers or adhesive tape to the instrument's tube. The tube may tear.
- Drain the water in the water pipe for about 2 minutes before starting treatment every day.
- When the equipment is not in use, wash or drain the water and air pipes.
- Thoroughly disinfect the equipment.
- After treatment, water the spittoon several times.
- Be careful when treating patients for a long time as pressure sores may occur.
- When replacing the tips or bars of all instruments, be sure to replace them in a stopped state. Replacing during operation may cause unexpected accidents and hazards. In addition, replacing with a non-certified product may cause unexpected accidents or hazards.
- When inserting each instrument into the holder, make sure to tighten it slowly and securely.
- Do not subject the watering system to a serious impact or load.
- Do not overuse or disassemble the equipment.
- If you reach out to a table or instrument, your hand or sleeve may become caught on the instrument in the holder, resulting in injury or infection.
- Be careful not to let your body come into contact with or around the light head during or after use.
- Do not look directly at the lamp while it is on or shine it directly into the patient's eyes.

Electrical Precautions

- Be sure to use 220V for input power.
- Do not install the equipment in a place where it is difficult to connect the power plug.
- If you need to change the input power, contact the nearest sales office or manufacturer.
- Do not unplug the power plug forcibly.
- Periodically wipe off foreign substances (dust, water, etc.) on the power plug terminals and contacts with a dry cloth.
- Do not forcibly bend, pull, twist, or tie the power cord. Also, do not hang it on metal parts, place heavy objects on it, insert it between instruments, or push it behind the product.
- Connect the power plug all the way to the outlet, and do not use a damaged power plug, power cord, or loose outlet.
- To disconnect power from the equipment, the power cord must be disconnected from the instrument.

Warning

- Do not pour or spray liquid substances such as water or sprays on this product. It may cause electric shock, damage to equipment, fire, etc.
- Do not store flammable materials around the equipment, otherwise it may cause a fire
- Never operate the operation switch of the equipment while the equipment is operating. It may cause equipment malfunction.
- In the event of an electrical fire, a fire extinguisher marked for that purpose must be used. The use of water or liquids in electrical fires can cause serious damage and damage to people and equipment. Disconnect the electrical plug before extinguishing the fire to avoid the risk of electric shock.
- Do not sterilize or disinfect with a spray. The sprayed sterilizer/disinfectant may ignite.

-

Manufacturer's Obligation Policy

For Dentium's bright Chair 2 dental Unit and Chair (ICT-CH02S), the manufacturer is responsible for product safety and normal operation only in the following cases.

- When it is not an accident or equipment damage caused by the customer's negligence
- When the product is used normally in accordance with the user manual

- When using a genuine product approved by the manufacturer
- In case of maintenance/maintenance by an authorized dealer in the manufacturing
- When the product is installed by a dealer authorized by the manufacturer
- When the equipment is installed in compliance with the installation conditions and precautions presented by the manufacturer

	- Never disassemble or modify the product. - An untrained person may cause harm when replacing parts
---	---

Safety instructions

Symbols used in this manual


- In this manual, for the efficient use of the product, the following symbols are used to fully understand the meaning of the symbols and follow them.
- To avoid personal injury and property damage, observe the safety precautions specified in this document. Notices are indicated by the symbol below.
-

Symbol	Name	Description
	CAUTION	Failure to comply may result in user injury.
	WARNING	Failure to comply may result in death or serious injury to the user.
	ELECTRIC SHOCK	If touched, there is a risk of injury from electric shock.
IPX1	RATE OF WATERPROOF	Shows waterproof rating of 1.
	ELECTROMAGNETIC INTERFERENCE	Indicates electromagnetic interference.
	SEPERATE DISPOSAL OF ELECTRIC AND ELECTRONOC PRODUCT	It must be disposed of as instructed.
	EMERGENCY SWITCH	Display the emergency switch.

	ON	Shows power on.
	OFF	Shows power off.
	PE	Indicates the protective earth terminal (safety earth).
	NOTE	The operating instructions in the user manual must be followed.
	Applied part (Protection Against Electric Shock)	The degree of protection against electric shock is type B.
	MANUFACTURING DATE	Displays the date of manufacture.
	MANUFACTURER	Identifies the manufacturer of the product.
	EUROPEAN REPRESENTATIVE	Represents a European representative.
	MODEL NAME	Indicates the manufacturer's model name.
	Medical Device	Indicates that the product is a medical device
	UDI Symbol	Indicates the UDI of the product
	SERIAL NO.	Indicates the manufacturer's serial number.
	ACTIVE POWER	Displays power consumption.
	Temperature limit	Indicates the storage temperature range.
	Humidity limitation	Indicates the storage humidity range.

2. Name of Each Part


2.1 bright Chair 2 (ICT-CH02S)

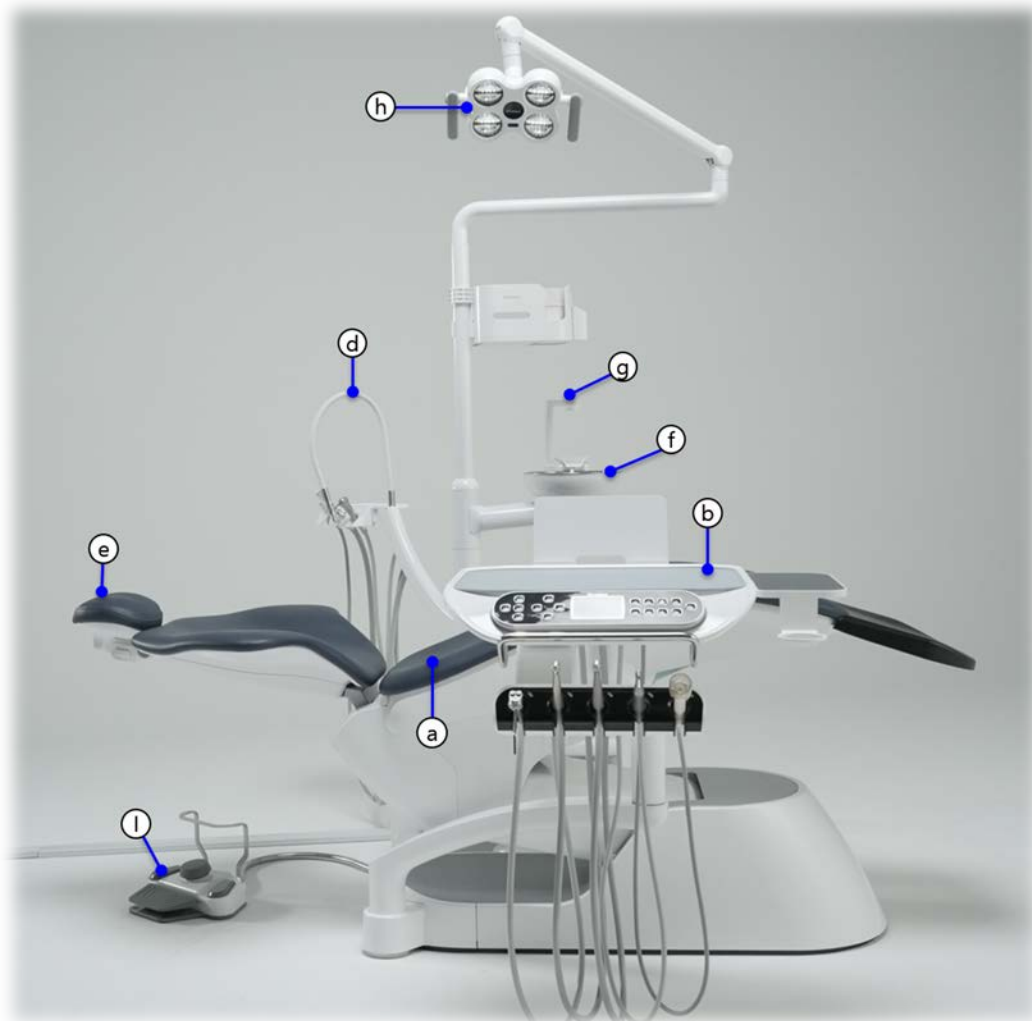

No.	Name	Function
(a)	Seat	Move the patient to the treatment location.
(b)	Dr. table	Operate the unit chair and mount the medical equipment used by the doctor.
(c)	Auxiliary table	Operate the unit chair and mount the medical equipment used by the nurse
(d)	Auxiliary Suction	Auxiliary Suction Module for One-Person Treatment
(e)	Headrest	Move the patient's head to the treatment position.
(f)	Spittoon	Patient can gargle.
(g)	Water inlet	Supply water for gargling.
(h)	Dental light	The LED light is irradiated to the patient's mouth for treatment.
(i)	Foot controller	Controls the functions required for chair operation.

3. How to operate each part

3.1 How to use the power system

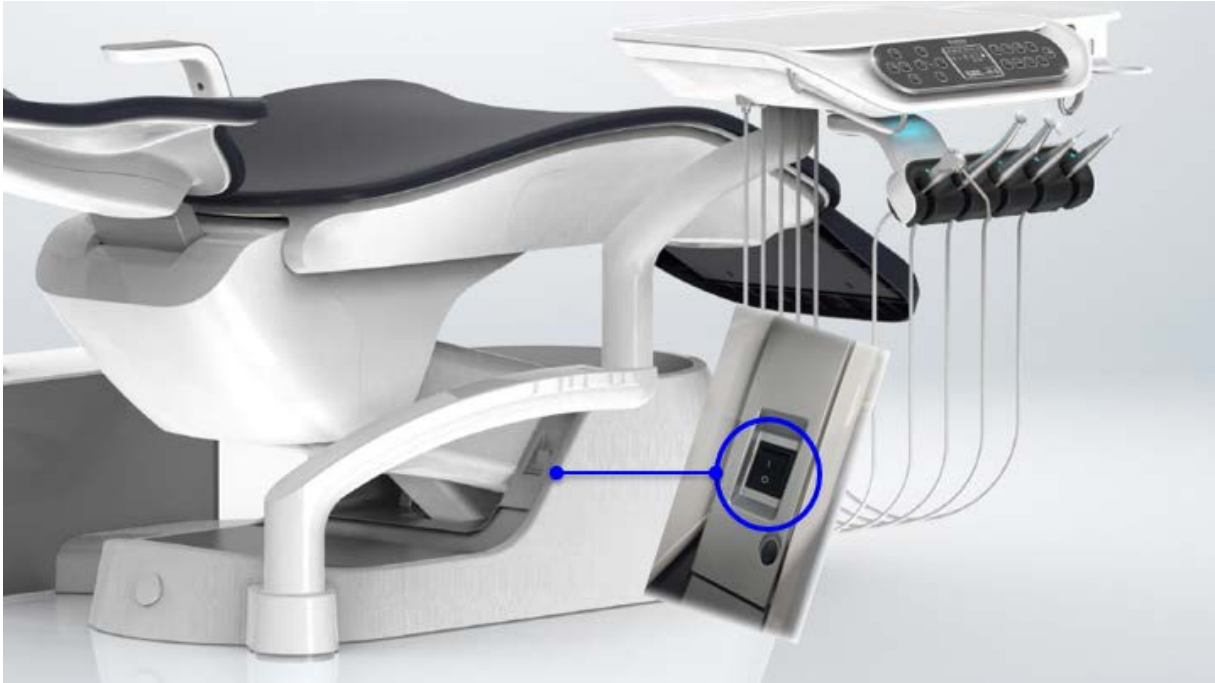

Name	How to use
Main switch	Turn on the power by pressing the power switch to the "I" direction. I : Power on O : Power off

 Note	• Check and use the power supply (220[V]) for the device. • When the power is turned on, the doctor table LCD lights up.
---	--

	• When the device is not in use, the power switch must be turned OFF.
---	---

3.2 Dr. Table

3.2.1 How to use Dr. Table

- ◆ When the handpiece is taken out of the instrument holder, the corresponding instrument information is displayed on the LCD screen, and the handpiece operates when the foot switch is pressed.



No.	Name
(a)	3 way syringe
(b)	Two Air High Speed Handpieces
(c)	Control panel
(d)	Air Low speed handpiece & Electric Low speed handpiece motor
(e)	Scaler
(f)	Mouse tray

 Warning	• Do not place objects more than 3 kg on the table. (with accessories) • Do not use excessive force or subject it to strong impact.
--	---

 Note	• The order of instrument placement is fixed. • Dr. Table's silicone pad sterilization conditions: up to 135 degrees for 3 minutes or more, within 20 minutes
---	---

a. How to use the control panel



Button	How to use
	When the button is pressed, the medical chair rises, and when released, it stops.
	When the button is pressed, the treatment chair descends, and when released, it stops.
	When the button is pressed, the back of the treatment chair rises, and when released, it stops.
	When the button is pressed, the back of the treatment chair descends, and when released, it stops.
	1. Press the button to move to the basic position. 2. If the dental light is on while the chair is moving automatically, it is automatically turned off. 3. After moving to the basic position, the dental light does not turn on. 4. When moving to the basic position is completed using the basic position button, press the button to move to the previous treatment position. Upon completion of the movement, the dental light turns on automatically.
	1. Navigate to the saved location. 2. To save the position, after setting the chair position, select button 1, 2, or 3 and press for more than 3 seconds to save the position. 3. If pressed again during movement, it stops. 4. When moving to the saved position is completed, the dental light turns on automatically.
	
	




Warning

- Be careful not to get your feet caught in the lower part while moving the chair.
- After detecting being caught in the lower part while moving the chair, it rises to a certain height and then stops.


Buttons	How to use																								
	1. You can control the water supply On/Off by holding the handpiece and pressing the button. 2. If you press the button without holding the handpiece, you can adjust the amount of water in the cuspidor. After changing the setting value, press the watering button to save it. <table><tr><th>Button</th><th>Display</th><th>Cup awareness</th><th>Water control</th><th>Setting</th></tr><tr><td rowspan="4">F/R</td><td>AS</td><td>Auto</td><td>Sensor</td><td rowspan="4">1~10</td></tr><tr><td>At</td><td>Auto</td><td>Time</td></tr><tr><td>HS</td><td>By hand</td><td>Sensor</td></tr><tr><td>Ht</td><td>By hand</td><td>Time</td></tr><tr><td>MEM</td><td>SP</td><td colspan="2">Spitton water out time</td><td>1~20</td></tr></table> 3. Press the button for more than 3 seconds to enter the water pipe cleaning mode.	Button	Display	Cup awareness	Water control	Setting	F/R	AS	Auto	Sensor	1~10	At	Auto	Time	HS	By hand	Sensor	Ht	By hand	Time	MEM	SP	Spitton water out time		1~20
Button	Display	Cup awareness	Water control	Setting																					
F/R	AS	Auto	Sensor	1~10																					
	At	Auto	Time																						
	HS	By hand	Sensor																						
	Ht	By hand	Time																						
MEM	SP	Spitton water out time		1~20																					
	1. You can turn On / Off the handpiece light by holding the handpiece and pressing the button. 2. You can adjust the light brightness of each handpiece by pressing the button without lifting the handpiece. After changing the setting value, press the light key to save it. <table><tr><th>Button</th><th>Display</th><th>Handpiece Type</th><th>Setting</th></tr><tr><td rowspan="4">F/R</td><td>HP1</td><td>Air high speed handpiece1</td><td>1~10</td></tr><tr><td>HP2</td><td>Air high speed handpiece 2</td><td>1~10</td></tr><tr><td>LP/EL</td><td>Air low speed handpiece / Electric low speed handpiece</td><td>1~10</td></tr><tr><td>SC</td><td>Scaler</td><td>1~10</td></tr></table>	Button	Display	Handpiece Type	Setting	F/R	HP1	Air high speed handpiece1	1~10	HP2	Air high speed handpiece 2	1~10	LP/EL	Air low speed handpiece / Electric low speed handpiece	1~10	SC	Scaler	1~10							
Button	Display	Handpiece Type	Setting																						
F/R	HP1	Air high speed handpiece1	1~10																						
	HP2	Air high speed handpiece 2	1~10																						
	LP/EL	Air low speed handpiece / Electric low speed handpiece	1~10																						
	SC	Scaler	1~10																						
	You can turn On / Off the dental light. Press once to turn on, press again to turn off.																								
	1. Rotation direction can be set when using an electric motor – F : Forward, R : Reverse 2. You can set the mode when using the scaler. – Operates in Scaling Mode after power on. – When you press the button and 1 is displayed on the screen, Perio																								

	Mode ((NSK only) - If you press the button again and 2 is displayed on the screen, it is Endo Mode. ((NSK only) - If the set value is not saved even if the power of the product is turned off and then turned on, it operates in the default scaling mode.
	1. electric motor rpm can be stored in 3 memories. - If you press Mem once, it blinks M1, and if you press it again, M2 and M3 blink sequentially. - At this time, press the right button to set the RPM.( ) - If the Mem button is pressed for 3 seconds or there is no input for more than 5 seconds, it is automatically saved.
	1. Press the button to start the set alarm - How to set the alarm time is as follows. -   Press the button to set the time - Can be set up to 99 minutes and 99 seconds - If there is no input, the alarm starts automatically after 3 seconds at the set time. - After changing, pressing the alarm button again cancels the alarm setting change and turns it off.
	1. Electric motor rpm and alarm time can be changed (rising)
	1. Electric motor rpm and alarm time can be changed (falling edge)
	1. Warmer can be turned on/off - The warmer is automatically turned off if there is no operation for 30 minutes. - (chair position manipulation, handpiece motion, suction motion, cuspidor motion) - The warmer turns on automatically if there is any operation while the image is displayed.()

b LCD Description

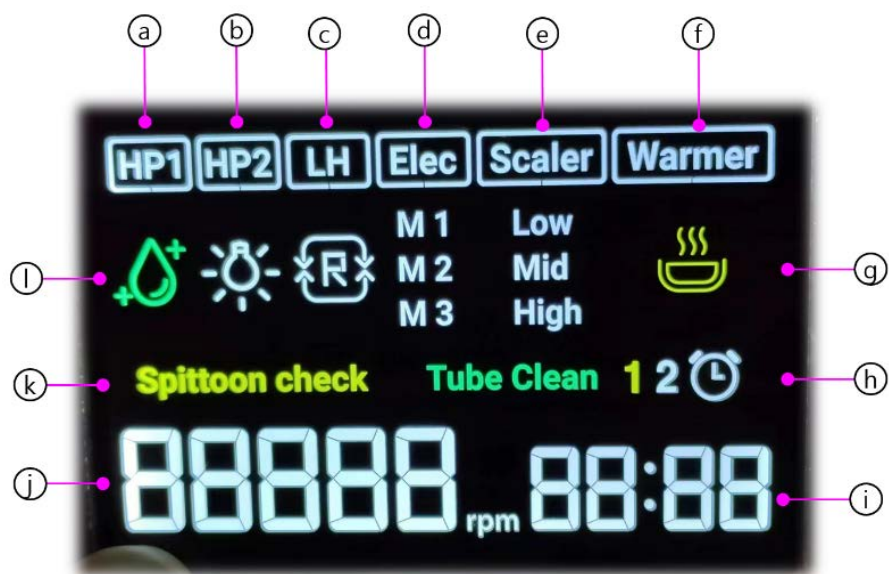

No.	Image	Description
HP1 HP2		- Displays high handpieces 1 and 2. - When the handpiece is removed from the holder, the border of HP1 or HP2 on the LCD top screen blinks to indicate selection.
LH		- Shows the low handpiece. - When the handpiece is removed from the holder, the border of LH on the top screen of the LCD flickers to indicate selection. (This mark is displayed when air handpiece is installed)
Elec		- Display the electric motor - When the handpiece is removed from the holder, the Elec border on the LCD top screen blinks to indicate selection. (This function is displayed when an electric motor is installed)
Scaler		- Display Cavitron selection - When the handpiece is removed from the holder, the border of the scaler on the top right of the LCD blinks to indicate selection. - You can select the intensity from 1 to 10 (Adjust the intensity using the up/down buttons on the right control panel)

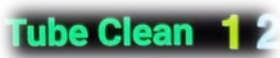Warmer		1. Display the warmer - You can select ON/OFF through the switch - If there is no operation in the displayed state, the warmer turns off after 30 minutes.
g		1. Displays warmer operation status 2. Icon display state is ON, OFF when off.
h		1. Displays the water pipe washing function. Press the irrigation button for more than 3 seconds to change to the water pipe washing mode. For details, refer to 4.5 Water pipe washing mode. 2. Also used for scaler mode indication. ()
i		1. Displays the alarm function. Press the alarm button to activate the alarm function at the bottom right. When OFF, press the alarm button again to turn it off.
j		1. The display at the bottom left of the LCD indicates the electric motor rpm or the output status of the handpiece and the set strength of the scaler.
k		1. Spittoon displays rotation detection status, and chair motion is restricted while displayed.
l		1. Displays the irrigation status. ON/OFF can be selected.
m		1. Displays the handpiece optical operation status. ON/OFF can be selected through the switch.
n		1. Displays the forward/reverse direction of the electric motor. A switch allows you to choose between 'F' for forward and 'R' for reverse rotation direction.
o		1. Displayed when a jamming occurs at the lower part of the chair.




Note

- It is not possible to move the position of the medical chair while the handpiece is in operation.
- When using multiple handpieces, the first detachable handpiece is activated.

3.2.2 How to adjust the water quantity of the handpiece

- ◆ There are 5 volume control valves for each handpiece as shown in the figure below. Turning it clockwise decreases the watering amount, and turning it counterclockwise increases the watering amount. You can adjust the watering amount by adjusting the water amount.(It's on the bottom right of the doctor table)

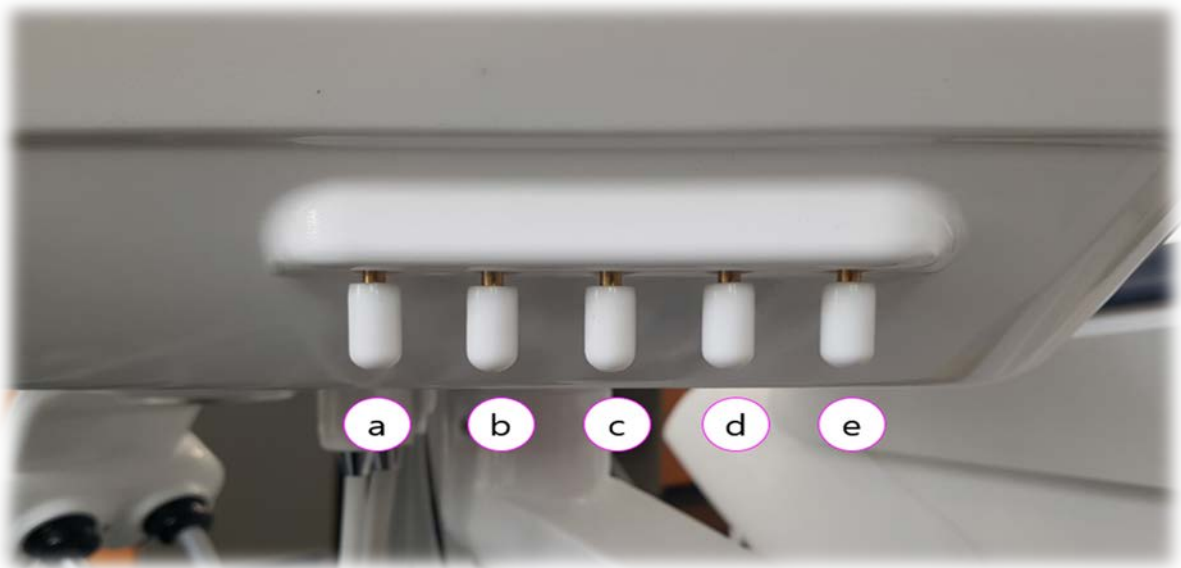

No.	Name
(a)	3 way syringe
(b)	High Speed Handpiece 1
(c)	High Speed Handpiece 2
(d)	Low speed handpiece
(e)	Scaler

3.2.3 How to use the high handpiece

- ◆ b After removing the handpiece from the holder, press the foot switch pedal to operate.
- ◆ Speed control can be adjusted according to the pressing depth of the foot switch.



Display in screen	How to use
	When using the High Handpiece, the High Handpiece screen appears on the LCD screen. Press the function button of the front panel to change settings.

Display in screen	Button	Function
		Press the button to select Handpiece Light On/Off.
		Press the button to select Irrigation On/Off

 Warning	- You can't change settings while the handpiece is operating. - Lifting the handpiece while pressing the foot control switch may result in personal injury. Be sure to hold the handpiece first and, then press the foot switch.
 Note	The position of the medical chair cannot be moved while the handpiece is in operation.

3.2.4 How to use the Low Handpiece

- ◆ Remove the handpiece from the holder and press the foot switch pedal to operate.
- ◆ The speed can be adjusted according to the depth of pressing of the foot switch.


Display in screen	How to use
	When the low handpiece is removed, the LH window on the home screen flashes to indicate the selection. You can change the setting by pressing the function button on the panel located on the front of Dr. Table.

Display in screen	Button	Function
		Press the button to select Handpiece Light On/Off.
		Press the button to select Irrigation On/Off

 Warning	- You can't change settings while the handpiece is operating. - Lifting the handpiece while pressing the foot control switch may result in personal injury. Be sure to hold the handpiece first and, then press the foot switch.
--	---

 Note	The position of the medical chair cannot be moved while the handpiece is in operation.
---	--

3.2.5 How to use Electric Motor (Option ICT-BE1)

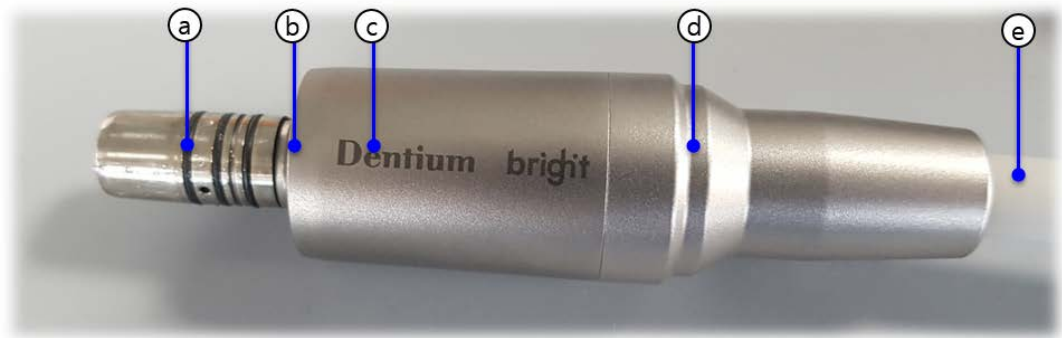

NO.	NAME
(a)	O-ring
(b)	Optical LED
(c)	Motor Body
(d)	Motor Connector
(e)	Motor Tubing (8.5mm x 2,000mm)



- ◆ After removing the handpiece from the holder, press the foot switch pedal to operate.
- ◆ The speed can be adjusted according to the depth of pressing of the foot switch.

Display in screen	How to use
	When you take out the electro handpiece, the LCD screen on the top shows the Elec (electro handpiece) setting screen. You can change the RPM by adjusting the UP/DOWN arrow buttons on the front of the doctor table

Display in screen	Button	Function
		1. Press the "MEM" button to use the RPMs stored in each memory. Press the switch to move M1, M2, M3 in order. - Press arrow for 3 seconds after RPM change to the selected memory to save.
		1. RPM increase: press up arrow() 2. RPM decrease: press down arrow() - It can be set from 3,000 to 40,000 RPM.
		1. "F" Rotate clockwise. 2. "R" Rotate counterclockwise.
		1. Press the button to set the Irrigation On/Off
		1. Press the button to control the Handpiece Light On/Off.

 Warning	• When the handpiece is in operation, other operation switch functions do not change. • Using the handpiece while pressing the foot control can cause injury. Be sure to hold the handpiece and press the foot switch.
--	---

	- Using the handpiece while pressing the foot control can cause injury. Be sure to hold the handpiece and press the foot switch. Implantable systems such as pacemakers and implantable cardioverter defibrillators (ICDs) are subject to electric, magnetic, and electromagnetic fields. This motor system emits an electromagnetic field, which means there are some potential hazards. Interference may occur during use and malfunctions may occur. These devices can. Dentium will guide you as follows : - Check with patients and users for implanted devices before using this product - Please explain the situation to the patient and consider the risks. - Keep this product away from implanted devices. - Take appropriate first aid measures and take immediate action if the patient is ill. Symptom - An increased heart rate, irregular pulse, and lightheadedness may be signs of a heart problem. 4 Do not drop the motor. This may result in electrocution or damage leading to product malfunction. ◆ According to IEC 60601-1/ANSI/AAMI ES60601-1 the motor is potentially not suitable for use. - Explosive atmospheres or potentially explosive mixtures of anesthetic substances containing oxygen or nitrogen dioxide, where these gases may accumulate concentrated in confined spaces. The motor and handpiece tubing are not suitable for use in oxygen enriched atmospheres.
--	--

 Caution	- Follow the manufacturer's recommendations for electric motors and accessories. - A damaged motor poses a danger to the patient, user, and others. - bright elec products should only be used for dental care by qualified persons. - Instructions for use - Before each use, check the motor, motor tube and O-rings for damage, looseness and leaks. Make sure the motor tube is not twisted. Do not use the motor if damage is detected. - Do not look directly at the motor lighting as it may cause visual impairment. - Sterilize the motor before first use. - Be sure to supply cooling air to the motor. Otherwise, the surface temperature of the motor will rise to 60 degrees and may cause burns.
--	--

 Note	- While the handpiece is operating, the operation to move the medical chair cannot be performed. - Sterilization conditions are as follows. - High-pressure steam sterilization conditions: Maximum 135°C, 3 to 20 minutes
---	--

3.2.6 Usage environment and conditions

- ◆ Use only uncontaminated water and air.
- ◆ Proceed with regular function and maintenance checks.
- ◆ The product has an operating time of 3 minutes and provides an intermittent operating condition of 10 minutes off.
- ◆ If the surface temperature of the motor increases during continuous use, the output torque may decrease. Wait until it cools down sufficiently. When used, it can be used as normal output torque.

3.2.7 Periodic inspection

- ◆ The following items should be tested before treatment for safety and normal use.
 - Before using the motor and tube, make sure they are working properly.
 - Always ensure that cooling is delivered to the appropriate motors via air and coolant. If the cooling water supply fails, the motor should be stopped immediately.
 - Check the motor, motor tube and O-rings for damage, looseness, leaks and biofilm before each use. Do not use the motor or motor tube if the motor tube is kinked or damaged. Requires use after service or replacement.

3.2.8 How to connect and disconnect the motor and Handpiece

- ◆ To connect the motor and handpiece, align the grooves as shown in the image above and turn until it clicks.
- ◆ Pull the handpiece to separate



3.2.9 Cleaning and disinfection methods

- ◆ Clean and disinfect the motor and motor tube immediately after examining a patient.
- ◆ Follow local instructions, standards and instructions for cleaning, disinfection and sterilization.
 - Do not use disinfectants containing chlorine or protein fixatives.
 - Do not allow liquids to enter the motor during cleaning and disinfection.
 - Do not use a thermal washer disinfectant to clean the motor or motor tube.
 - Do not lubricate permanently lubricated motors.
- ◆ Disinfection method
 - Wipe the outside of the motor and motor tube with a clean cloth moistened with disinfectant or 60-90% ethanol.
 - Chlorine or protein fixatives certified by duly certified organs. Follow the disinfectant manufacturer's specifications for correct use.




Warning

- Do not use autoclave sterilizers that use rapid heating or cooling. Sudden temperature changes cause component degradation and failure.
- Do not use sterilizer above 277°F (136°C) including drying process. It may be the cause.

	• The product is hot after sterilization and should not be touched immediately. Doing so may cause burns. • Be careful not to press the footswitch when removing the motor and handpiece. Failure to do so may result in motor failure or injury.
--	--

 Caution	• Follow local directives, standards and instructions.: - Cleaning and disinfecting the motor before sterilizing it ensures that the sterilization process is efficient and prevents damage to the motor. - Do not sterilize motor tubes. - Remove only dry, sterile products from the sterilizer. - Store sterile products in a dry, dust-free environment. - Sterilize the motor with USFDA approved paper/foil laminate sterilization wrap per ISO 11607-1. - If the sterilization instruments contain disinfectants or other residues, the use of chemicals can discolor the surface of the motor and damage internal components. - Motors are recommended for autoclave sterilization. Other sterilization methods have not been validated. - Handle the motors carefully to avoid damaging the front LEDs.
--	--

3.2.10 Sterilization method

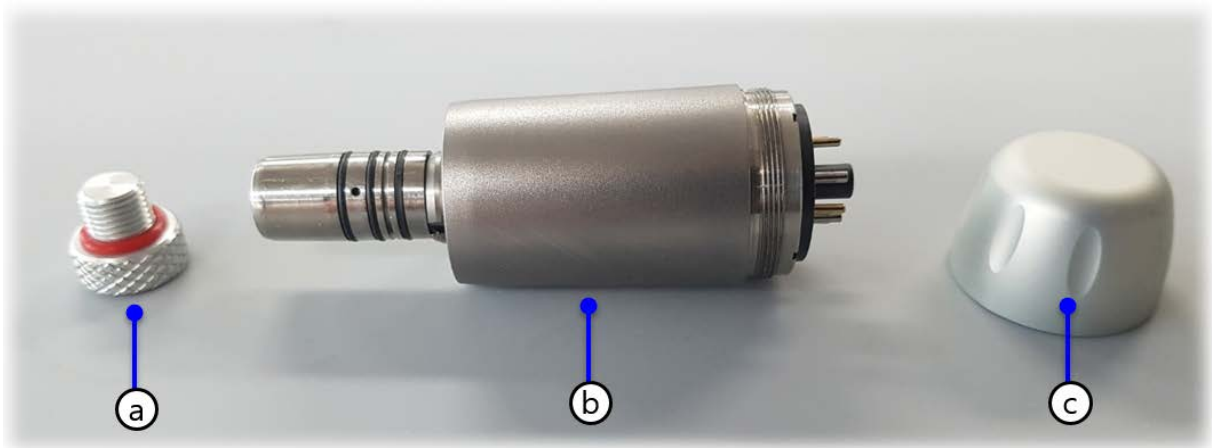

NO.	NAME
①	Sterile stopper 1
②	Motor
③	Sterile stopper 2

1. Clean and disinfect the motor before sterilizing.
2. Tighten the sterilization stoppers a and c in the figure above.
3. Seal the motor with a USFDA approved paper/foil sterilization pouch per ISO 11607-1.
4. The sterilization conditions for sterilizing the motor at 135°C for the following hours are as follows:

- **High-pressure steam sterilization conditions: Maximum 135°C, 3 to 20 minutes**

5. Allow to dry for approximately 30 minutes or more before removing from the sterilizer. No water droplets should be visible on the sterilization pouch before the load is completely dry and removed from the sterilizer. When storing the motor, keep it clean and dry in the sterile pouch.

(Refer to your local medical device regulations for more information on sterilization standards.)

3.2.11 Maintenance

 Warning	• Failure to turn off the power before performing maintenance may result in product damage or malfunction.
--	--

1. Electric motors must be maintained according to infection control according to this


instruction.

2. Use of parts not approved by Dentium voids all claims under warranty and all other claims.
3. Regular servicing of the motor, including accessories, is required for proper function and safety.
4. Damaged O-rings may cause the following symptoms Please contact your Dentium dealer or customer service for replacement of damaged O-rings.
 - 4.1 Air/water leaks.
 - 4.2 Air/water cannot reach the inside of the handpiece, so it will not work properly.
 - 4.3 Vibration.
 - 4.4 Difficult to connect/disconnect to handpiece.

3.2.12 How to use the Scaler(bright Scaler _ ICT-BS1)



a. Main components

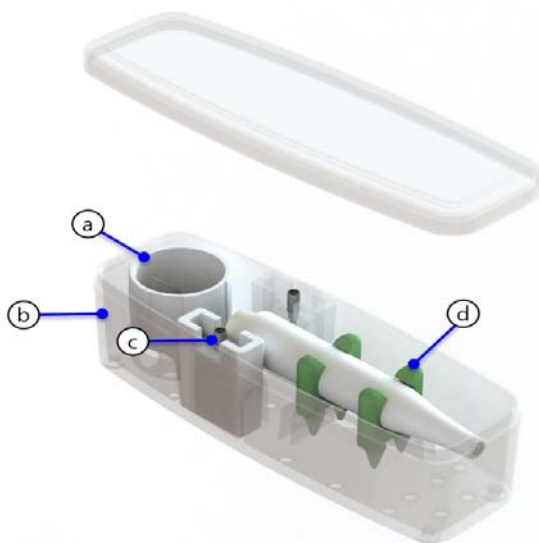

NO.	NAME
①	Torque wrench
②	Sterile case
③	Scaler tip(A, P, PS)
④	Scaler body fixed part

 Note	• When using high-pressure steam sterilization, please refer to the following • High-pressure steam sterilization conditions: up to 135 ° C for 3 minutes or more and within 20 minutes - ※ When sterilization conditions are different from the above, use according to the - instructions of the sterilizer manufacturer.
---	--

- ◆ After removing the scaler handpiece from the holder, press the foot switch pedal to operate.
- ◆ Power can be adjusted according to the scaler adjustment on the LCD display screen. (1 to 10 steps)

Display in screen	How to use
	- When the scaler is taken out of the holder, the LCD Scaler outline blinks.

Display in screen	Button	Function
		1. Adjust the intensity with the scaler removed from the holder. - You can adjust the level by pressing the arrow on the left (1 to 10 levels adjustable)
		1. Press the button to set the Irrigation On/Off
		1. Press the button to control the Handpiece Light On/Off.

 Warning	• Be careful since you can't check the operating status of the scaler when using the One-Touch function. • You can't change the settings while the handpiece is in operation.
 Note	• The position of the medical chair cannot be moved while the handpiece is in operation. • One-Touch Function: If you press this button when you use the scaler, it will continue operation (Option)

- How to use

◆ Precautions before use

- Secure power and water supply sources suitable for use.
- To prevent infection, be sure to sterilize the tip and handpiece in an autoclave before use.
- Use a torque wrench and tip holder when handling tips to prevent infection.
- Use handpieces, tips, tip holders, and sterilization boxes after sterilizing them in a high-pressure steam sterilizer.

◆ How to use

- Adjust vibration and water pressure appropriately using the adjustment panel on the dental treatment device.
- After selecting a tip suitable for the purpose, install it accurately using a torque wrench.
- Step on the foot pedal connected to the dental treatment device to check whether it works.
- Check if the watering is done properly during operation.
- In normal operation, the operator applies to the patient.

◆ Precautions for use

- Use only by a qualified dentist.
- Do not use for patients with cardiac pacemakers.
- Use for purposes other than intended is prohibited.
- Due to the risk of explosion, do not use in the presence of ignitable anesthetics or gases.
- Be sure to clean and sterilize handpieces and tips before and after use.
- Replace bent or deformed tips immediately, and replace tips worn by more than 2mm to avoid possible damage or injury during use.
- Be sure to use a torque wrench when attaching or detaching the tip.
(Installation and detachment using other tools may lead to reduced output and

shortened life of the device.)

- Be careful not to drop the device when using, cleaning, or sterilizing it. (vandalism)
- If the surface of the tip is peeled off, replace it with a new one after about 10 treatments.

◆ **Product maintenance and management**

Handpiece

- After each use, drive the handpiece with the tip removed to remove foreign substances remaining inside, then gently wipe the soiled surface with a soft brush or cloth.
- After daily use, wash with a corrosion inhibitor and rinse with water two or more times before storing.
- The handpiece is a consumable part and must be replaced when corroded or worn.

Scaler tip

- After each use, clean with a brush or ultrasonic cleaner using detergent.
- A minimum of three washes and rinses are required using a disposable syringe (minimum 50 ml) filled with deionized and distilled water.
- The tip is a consumable part and must be replaced when corroded or worn.

tube

- Wipe the tube with a soft cloth moistened with an alcohol-based liquid or neutral detergent.
- Dry the connector completely before connecting the handpiece.

Torque wrench

- The torque wrench is sterilized in a high-pressure steam sterilizer at 135°C for 3 minutes without a sterilization box and 20 minutes when sterilized in a sterilization box.

Storage method

- Store in a place away from water.
- Store in a place with low dust and moisture.
- Store away from direct sunlight.
- Temperature: -10°C ~ 40°C, Humidity: 10% ~ 95%, Atmospheric pressure: 500 ~ 1060hPa

Detailed tip description


- TIP-A : Used for all oral cavity
Used to remove general deposits
- TIP-P : Used for all oral cavity
Used for supragingival and sublingual scaling
- TIP-PS : Thin and long tip
Used on proximal surfaces and subgingival

※ **Caution**

- The tip is a consumable, and when the tip is worn by 1mm, the tartar removal efficiency is reduced by about 25%, and by about 50% when the tip is worn by 2mm. In addition, there is a concern that the vibration state may change due to wear and damage the tooth surface. Use the tip card to periodically check and manage the wear condition.

3.3 Auxiliary Table

3.3.1 Names of Parts

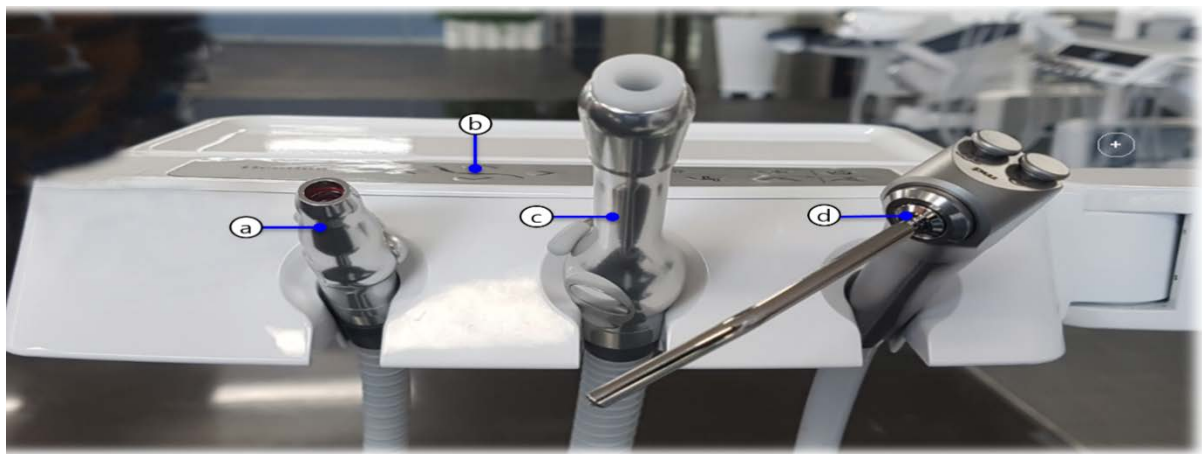

No.	Name
(a)	Suction source
(b)	Control panel
(c)	Suction stand
(d)	3-way syringe



Warning

- Do not put the load exceeding 2kg on the table (including accessories)
- Do not use excessive force or give strong impact.



Note

- You can mount the suction source on the auxiliary light.


3.3.2 How to use the auxiliary table



Button	Function	Button	Function
	The chair height rises when the button is pressed, and stops when released.		When the button is pressed, the chair back rises, and when it is released, it stops.
	Press the button to lower the chair height, and release to stop.		When the button is pressed, the chair climb descends and when released, it stops.
	After moving the chair to the position you want to save, press the button for more than 5 seconds to save the treatment position, and then press the button to automatically move to the saved position.		When the button is pressed, the chair moves to the start position.
			When the button is pressed, the dental light turns On / Off.
	Press the button to turn the auxiliary light On / Off.		Pressing the button turns the Suction Pump On / Off.
	Automatic water supply to the cuspidor : When the button is pressed and released within 2 seconds, it switches to automatic water supply mode and water is supplied for the set time. Manual water supply to the cuspidor : When the button is pressed for more than 2 seconds, the manual water supply mode is switched, and water is supplied		Cup automatic water supply : When the button is pressed and released within 2 seconds, it switches to automatic water supply mode, and water is supplied as much as the set amount Cup manual water supply : When the button is pressed for more than 2 seconds, the mode is converted to manual water supply mode, and water is supplied while the button is

	while the button is pressed.		pressed.
--	------------------------------	--	----------

3.3.3 Suction Syringe (Big/Small) How to use

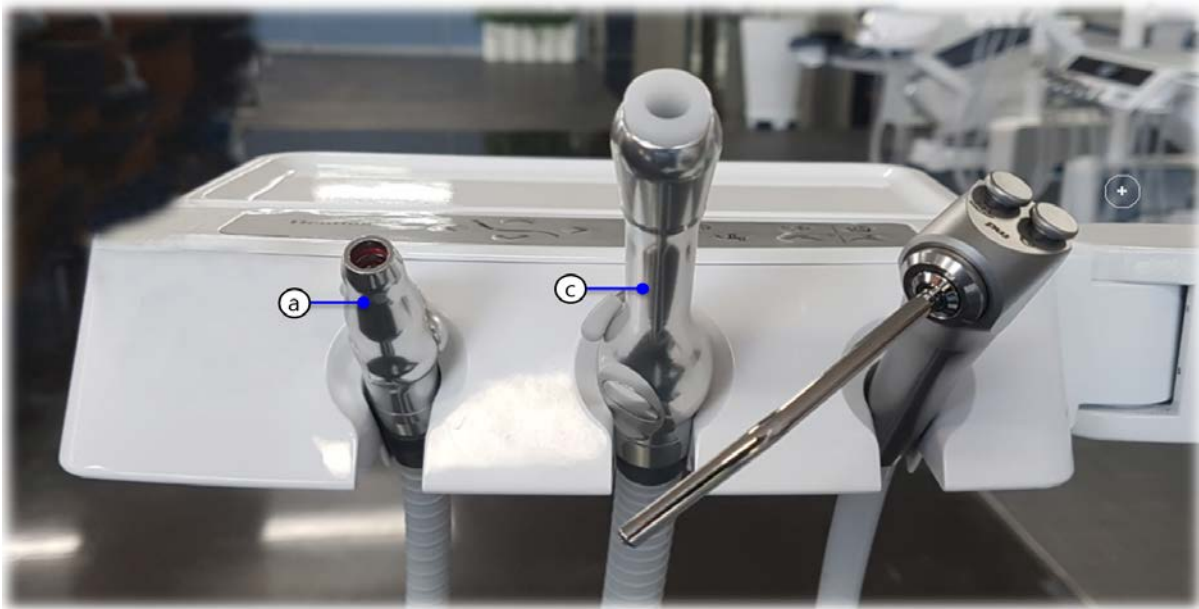

How to use
1. Remove the suction (big/Small) from the holder to use the suction. 2. Operate the locking lever up and down to absorb foreign substance.

Open (absorb)	Close (stop)
	
	



Note

- When one suction operates, both suctions operate. The suction force of each suction can be used independently using the locking lever.

3.3.4 3-Way Syringe How to use



- Product image



Name of Parts

NO.	Name
(a)	Syringe Tip
(b)	Syringe Body
(c)	Water button
(d)	Air button

Button	How to use
	Water is injected while the left button is pressed.
	Air blows while the right button is pressed
	Press the left and right buttons at the same time to use water spray

 Warning	Use only syringe tips provided by Dentium. If you use a syringe tip manufactured by Dentium, the syringe tip may pop out. Do not use third party products as tips may be swallowed or aspirated due to tip ejection.
--	--

3.3.5 How to remove Syringe Tip

- ◆ Pull the tip while pressing the push switch on the syringe tip as shown in the figure below. When tightening, assemble in reverse.



3.3.6 Cleaning and disinfection methods

- ◆ Clean and disinfect motors and motor tubes immediately after patient care.
- ◆ Follow local instructions, standards and instructions for cleaning, disinfection and sterilization.
 - Do not use disinfectants containing chlorine or protein fixatives.
 - Do not allow liquids to enter the motor during cleaning and disinfection.
 - Do not use a thermal washer disinfectant to clean the motor or motor tube.
- ◆ Disinfection method
 - Wipe the outside of the motor and motor tube with a clean cloth moistened with disinfectant or 60-90% ethanol.
 - Chlorine or protein fixatives certified by duly certified organs. Follow the disinfectant manufacturer's specifications for correct use.

 Warning	• Do not use autoclave sterilizers that use rapid heating or cooling. Sudden temperature changes cause component degradation and failure. • Do not use sterilizer above 277°F (136°C) including drying process. It may be the cause.. • The product is hot after sterilization and should not be touched immediately. Doing so may cause burns.
--	---

 Caution	※ Follow local guidelines, standards and guidelines. • Before sterilizing the motor, clean and disinfect it to prevent damage. • Do not sterilize the tubing of the 3-way syringe. • Remove only dry sterile products from the sterilizer. • Keep sterile products dry and free of dust. • Sterilize the motor with a USFDA-approved paper/foil sterilization cap conforming to ISO 11607-1. • If sterilized instruments contain disinfectants or other residues, use of chemicals may discolor the surface and damage internal components. • The use of automatic washers and heat sterilizers is not recommended as they may damage the syringe body and tip finish. Machining this type of metal may cause discoloration of the instrument.
--	--

3.3.7 Storage method

- Store in a dry, dust-free place. Do not store non-sterile.
- Refer to the sterile packaging manufacturer's instructions. (Used to determine how long sterility can be maintained.)

3.3.8 How to use single-person treatment suction

- It provides the convenience of one-person treatment. Through the exclusive suction module, you can use it conveniently as follows





- During practice, the suction module should be in the practice position as shown in the picture above.



- Pressing the "a" button activates the suction, allowing easy medical care without assistance. After use, please press the button again to turn off the suction.

3.5 Dental Light (bright light _ ICT-BL1)

Button	How to use
	Pressing the control module button of Dr. table and auxiliary table turns the dental light on.
	

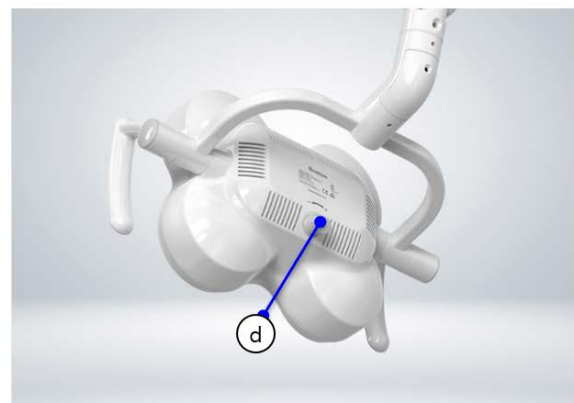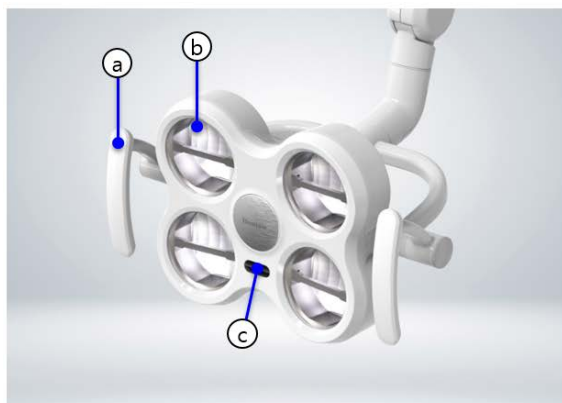


NO.	NAME	FUNCTION
(a)	Transport handle	Position adjustment knob
(b)	Optical module	Illustrator
(c)	Contactless power sensor	Contactless light power ON/OFF Motion Sensor (On/Off: sensing distance 50mm)
(d)	Lighting volume	Lighting brightness min & max

- ◆ The lighting distance is 600mm~800mm, and the maximum brightness at the center point is 10,000~50,000Lux or more.
- ◆ When the treatment chair moves to the saved position, the dental light is automatically turned on.
- Start and move the gargle position: Before moving the chair, the dental light is turned off

- Move to the previous position of gargle and chair memory 1, 2, 3: After chair movement is completed, the dental light turns on.
- ◆ Disinfect the dental light handle before use

3.5.1 Appearance

- You can freely adjust the up/down and left/right rotation angle of the dental light

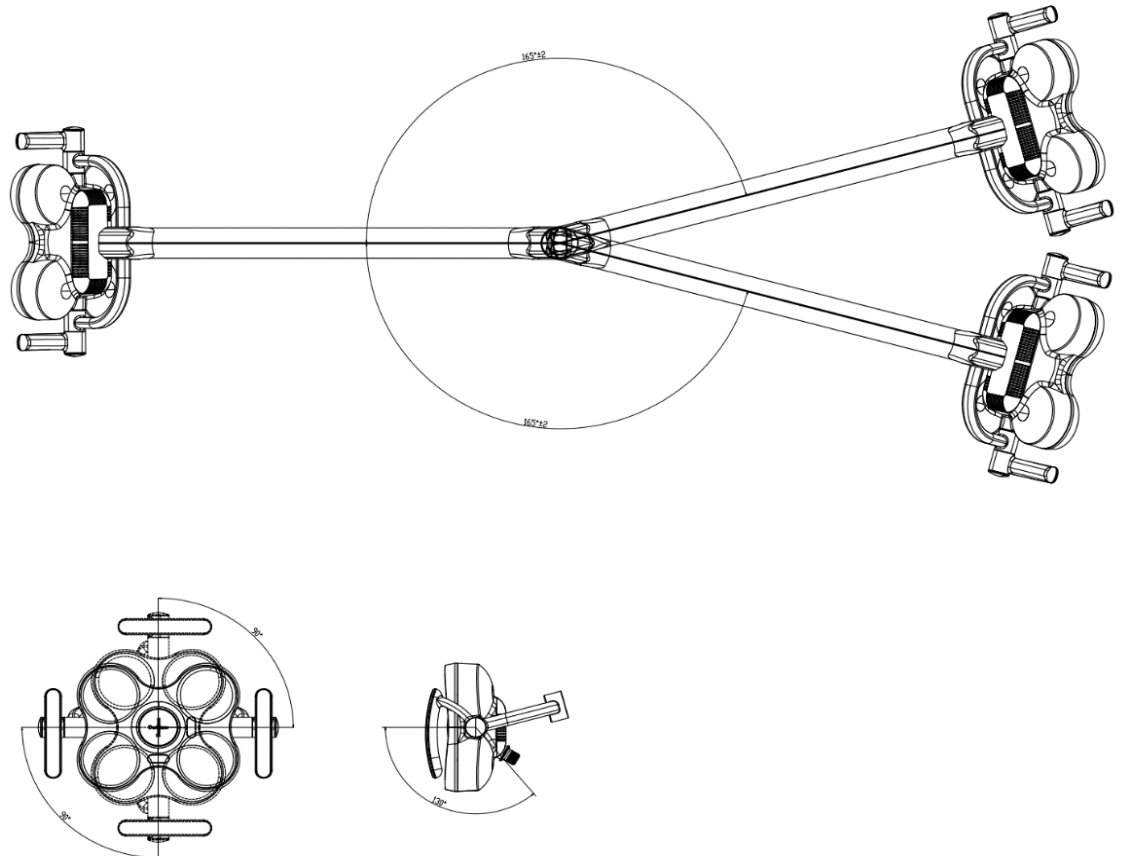

Warning

- When changing light positions, do not point the light at the patient, user, or third parties.
- When moving the light head, keep the light field away from the patient's eyes.



Caution

- Using third-party lighting may cause malfunction or failure
- Follow the manufacturer's instructions when using disinfectants to disinfect.
- Wipe when disinfecting.
- Do not immerse the product or parts in water or other liquids.

3.6 Watering device

- This is a device for spitting out and rinsing your mouth during and after treatment. The cup and spittoon can be watered at the same time. (Warm water is supplied depending on the option)



No.	Name	Purpose
Ⓐ	Water feeder	A unit that supplies a certain amount of water to a paper cup
Ⓑ	Sensor	A unit that recognizes the cup and controls the water level
Ⓒ	Cup stand	A unit that holds a paper cup
Ⓓ	Nozzle	A unit where the water to wash the spittoon is injected
Ⓔ	Spittoon	A place where the water rinsed the mouth is spitted. The batting table offers one of two materials (Plastic Ultem or Metal Stainless steel)

Automatic watering

- If you place a cup on the spittoon, the unit will automatically stop watering when a certain amount of water is filled. When you remove the water cup, the spittoon will stop the

watering automatically.

Button	How to use
	When you press and release the button on the auxiliary table within 2 seconds, it will switch to automatic watering mode and stop watering after watering the set amount.
	When you press and release the button on the auxiliary table within 2 seconds, it will switch to automatic watering mode and stop watering after watering the set amount.

Manual Watering

- Further, the cup and spittoon can be filled with water by operating the manual button on the auxiliary table.

Button	How to use
	If you press and hold the button on the auxiliary table for more than 2 seconds, it will switch to manual watering mode, and watering will proceed while the button is pressed.
	If you press and hold the button on the auxiliary table for more than 2 seconds, it will switch to manual watering mode, and watering will proceed while the button is pressed.

 Note	• For automatic cup watering, you can selectively set two modes: The quantity recognition mode and time control mode. To set this function, request the service to the local distributor. • The spittoon watering can be set as long as the user wants. To set this, request the service to the local distributor.
---	---

 Warning	• Please note that black, transparent, or metal cups are not recognized. • Please note that cups smaller than 2.5 ounces are not recognized.
--	---

- How to adjust the water quantity



When you open the front cover of the unit below, there are 3 volumes to control the water quantity.

Function	How to use
CUP	1. Water quantity increases : Rotate the switch counterclockwise 2. Water quantity decreases : Rotate the switch clockwise
SPITTOON	
3-Way SYRINGE	



3.7 Headrest

- The headrest can adjust the head up/down and angle according to the treatment situation as shown in the picture. 2 Joints can be adjusted, so it is convenient for treating children and the disabled.
- To adjust the headrest, press the head angle adjustment button to adjust the head to the desired position, then release the button to fix the headrest at that position. In addition, according to the height of the patient's upper body, you can hold the entire head and adjust it up/down to fix it in the desired position.

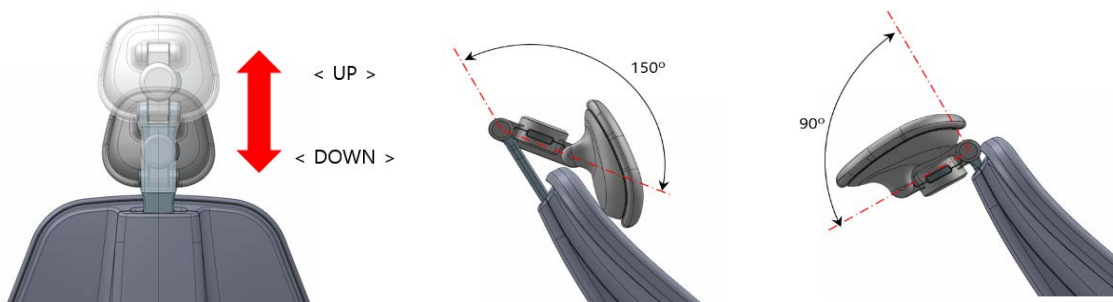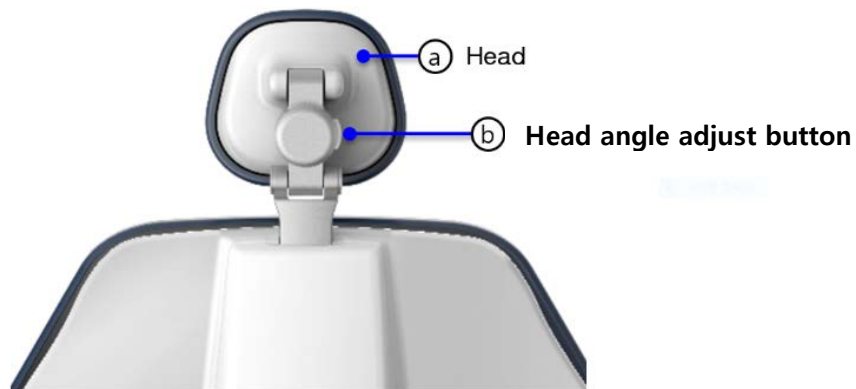


Warning

- When adjusting the head angle by using the Head Button, be careful as excessive angle adjustment may put a strain on the patient.
- When adjusting the head angle while the patient is lying down, be careful as an abrupt movement may occur due to the weight of the patient's head.
- If you press the headrest button while the scaler is running or the foot controller pedal is pressed, the system will not operate for safety.

3.8 Foot Controller

3.8.1 Functional foot controller

- The position of the medical chair can be controlled with the joystick and function buttons, and the handpiece can be operated using the pedal.

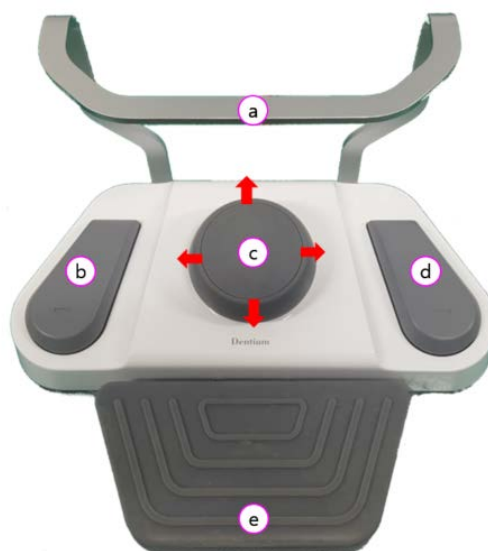

No.	Name	Description
①	Footrest	The foot switch is moved using a footrest
②	Function Button 1	Move to the starting position.
③	Joystick Up	When operating upward, the medical chair rises.
	Joystick Down	When operating downward, the medical chair goes down.
	Joystick Left	When operating to the left, the back of the medical chair does down.
	Joystick Right	When operating to the right, the back of the medical chair rises.
④	Function Button 2	Moves to the location of memory 1
⑤	Pedal	When you step on the pedal, the handpiece operates, and the speed of the handpiece increases or decreases according to the depth of pressing.




Warning

- If you operate the foot controller while moving the treatment chair, the chair will stop moving.

3.8.2 Disc type Foot Controller

- The position of the treatment chair can be controlled with the function button, and the handpiece can be operated using the pedal.

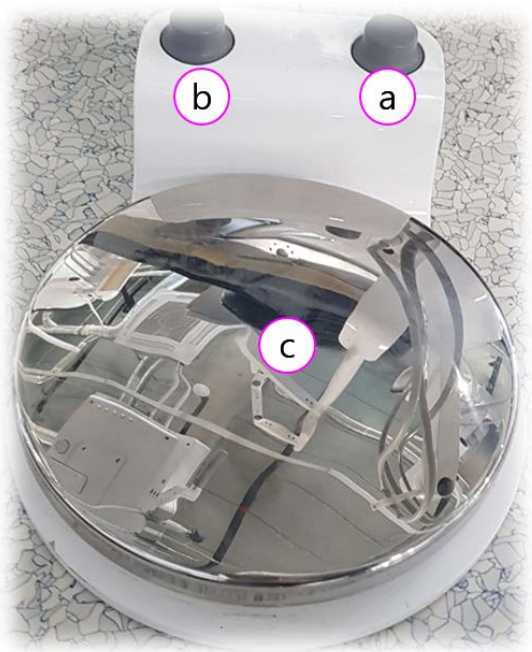

No.	Name	Description
Ⓐ	Function Button1	Moves to the start position
Ⓑ	Function Button2	Moves to the Memory 1 position
Ⓒ	Pedal	Stepping on the pedal operates the handpiece


 Warning	• If you operate the foot control while moving the chair, the chair stops moving.
--	---

 Note	- The function of the foot controller can be set variously in addition to setting the start position and memory 1 position. When changing to another setting, please contact the local distributor.
---	---

3.9 Emergency Operation

3.9.1 Guide for Operation

- In the event of an obstruction closing underneath the medical chair or an emergency situation, the medical chair will stop operating and automatically raise itself to protect the patient and system.

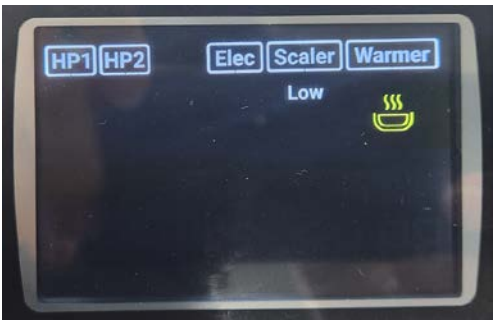

Name	Operating condition
Safety	• When an obstacle is detected at the lower part of the treatment chair, the operation of the treatment chair stops or rises to a specific position.

 Warning	• Always be careful not to let the patient, operator, or object enter the lower part of the treatment chair. • Do not arbitrarily operate the safety switch. • If the unit is damaged or does not work, contact our local distributor.
--	--

3.10 Cuspidor table rotation detection guidance

- A rotation detection sensor is applied to the ball hitting table. The chair will not work if it is twisted beyond a certain angle. In the case of medical treatment, please confirm the position of the batting table.




Situation	Operating conditions
Ball hitting table normal operation	There is no Spittoon check display on the LCD, and there is no problem with the movement of the chair and the seat plate.
Ball hitting table rotation detection	Chair climbing and seat functions stop LCD screen displays Spittoon check on the screen (Please align the positions of the batting table for use)

4. Maintenance

4.1 Replacement of Oil Can of Dr. table

- The oil can is located under Dr. table. When using the Air Turbine Handpiece, it collects backflow water and oil. It is located on the floor surface outside Dr. table.

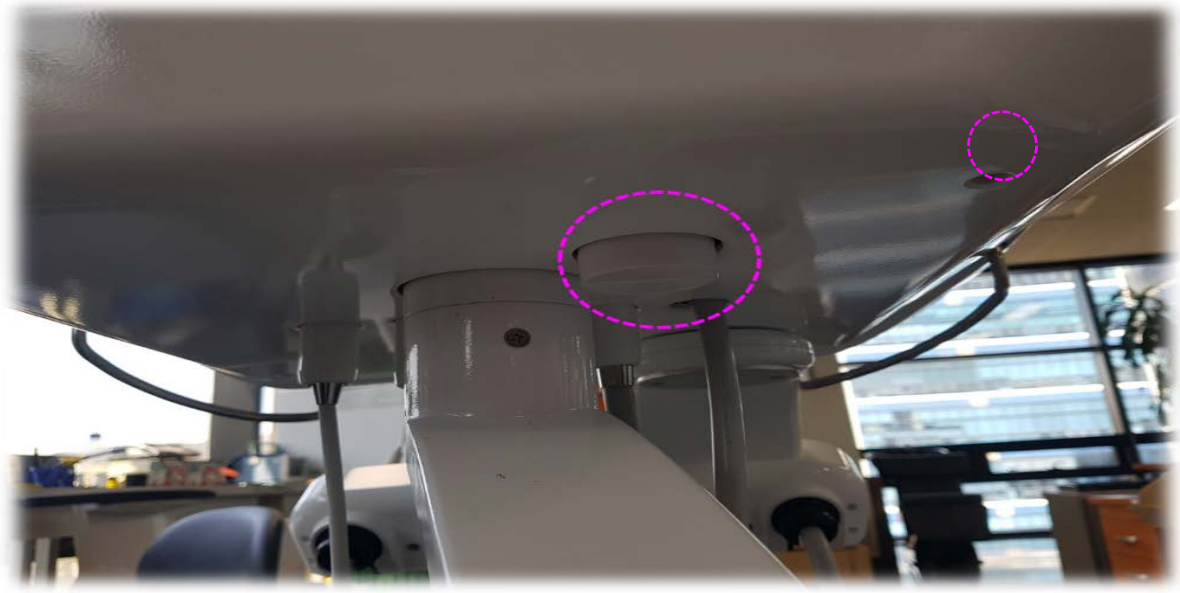

- Check the cleaning cycle every 6 months.
- The oil can is located at the bottom back of Dr. table, and when checking, hold the case and turn counterclockwise to disassemble it.
- After disassembling, remove the sponge inside and press it to separate the oil and reconnect in the reverse order.




Note

- If regular maintenance is not done, oil may fall to the floor.

4.2 How to clean the unit


- The parts that need periodic maintenance in the unit are the spittoon and the suction filter. You can prevent clogging of the drain due to foreign substances by cleaning the filter periodically.

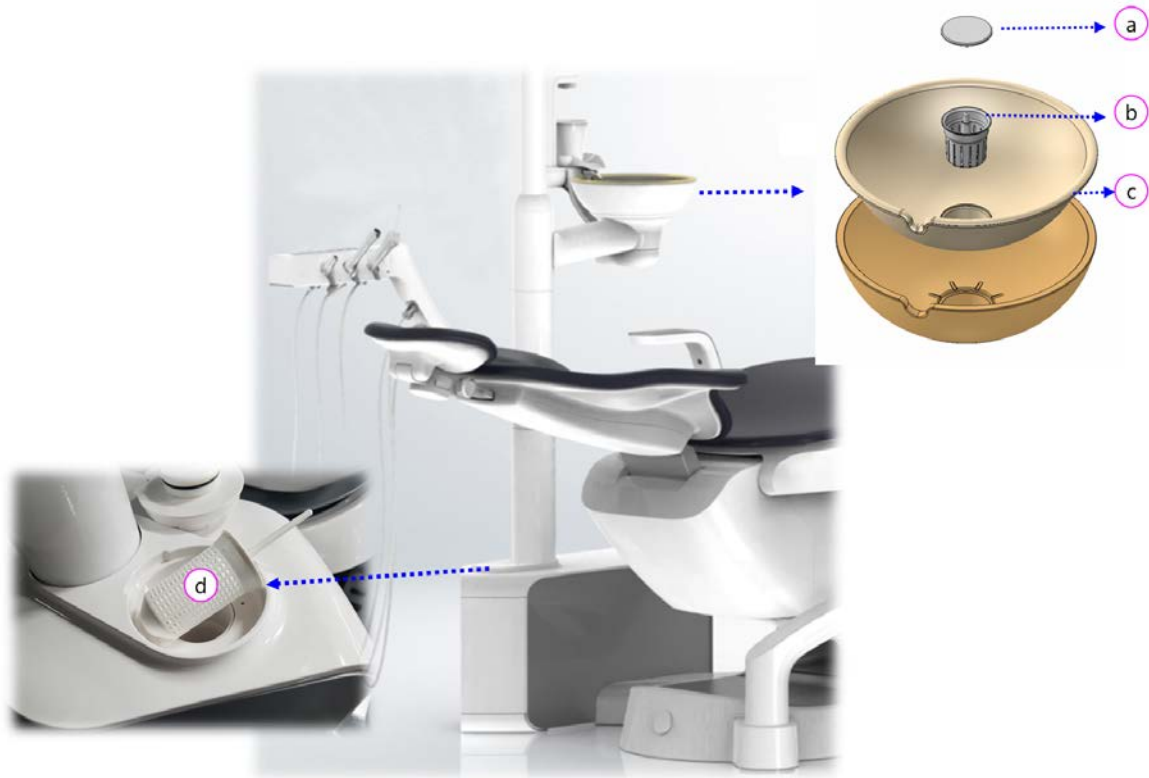

No	Cleaning area	Cleaning cycle
(a)	Spittoon filter plug	Once a day
(b)	Spittoon filter	Once a day
(c)	Spittoon	Once a day
(d)	Suction filter	Once a day

4.3 Cleaning the spittoon and spittoon filter

How to clean
After removing the cuspidor and lifting the filter cap, take out the filter to remove dirt and clean it with a soft brush.

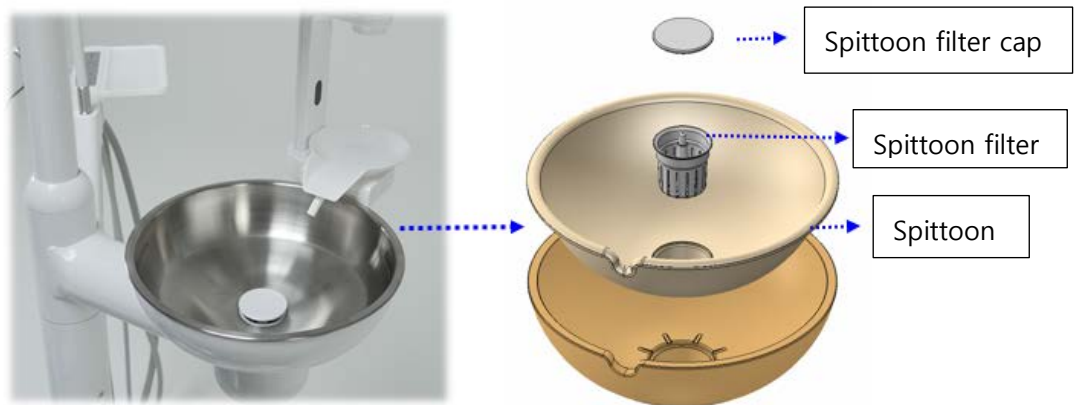

4.4 Cleaning Suction Filter

- Cleaning the suction filter can prevent the suction force from being weak.

How to clean
Open the cap in the direction of the arrow, remove the filter cap, remove dirt from the mesh, and wash it with a soft brush.



Note

- The spittoon can be sterilized by high pressure steam
- High-pressure steam sterilization conditions: up to 135°C, 3 minutes or more, within 20 minutes



Warning

- If you need to open the unit cover, please contact our local distributor.
- There is a risk of equipment failure or electric shock when operating arbitrarily.

4.5 Sanitation (Optional item)

- Sterilize all water pipes of the entire instrument.



Warning

- During sanitation, the system does not operate until the entire sanitation has been completed, so please perform the cleaning with care.




Note

- The sanitation is recommended to be done one a day.

- Preparations before sanitation

- Instrument setting

- ① Remove the cup stand of the spittoon and mount the sanitation cover.
- ② Remove instruments (except suction) of Dr. table and auxiliary table, and mount the sanitation referring to the following Fig.

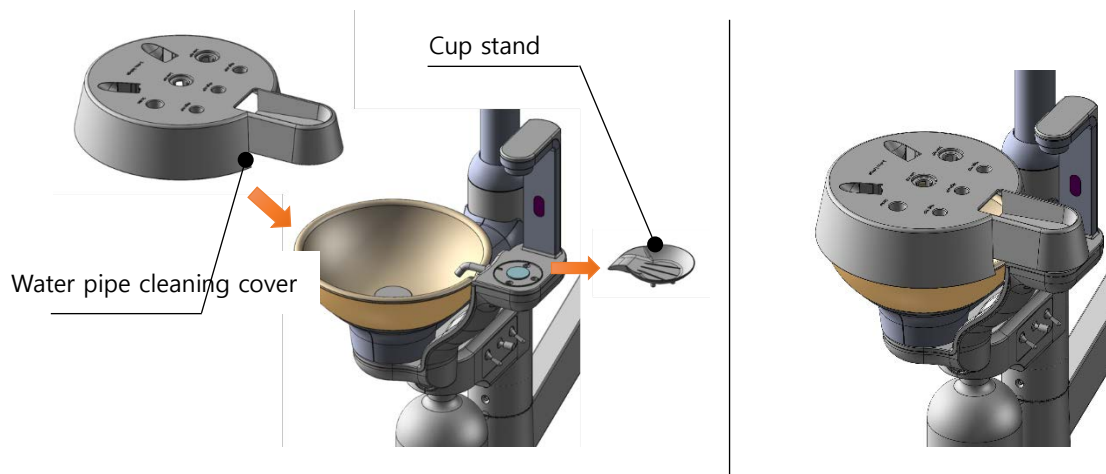

Setting method	
① Take out the handpieces one by one from Dr. table and place them on the sanitation cover. ② Refer to the table below for the location of the handpiece	

No	Item	Handpiece image
1	High H/P	
2	High H/P	
3	Air Low H/P	
4	Elec Low H/P	
5	Scaler	
6	3-Way Syringe	 


- Sanitation Cover and Mounting Image



Sanitation cover

Mounting image

- Distilled water + sanitation control module

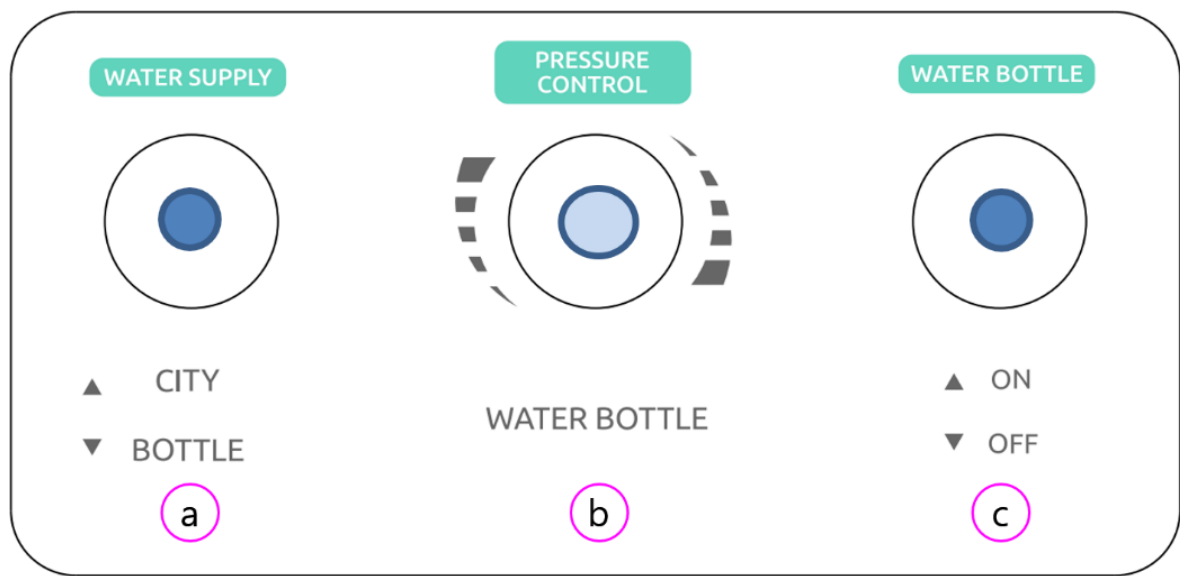

No	Name	Description
Ⓐ	WATER SUPPLY	Select the water to be used (water supply & bottle)
Ⓑ	PRESSURE CONTROL	Watering pressure volume (1.5~2Bar is recommended)
Ⓒ	WATER BOTTLE	Watering bottle pressure On/Off switch

- How to use the distilled water + water pipe cleaning control module

No	Operation mode	Setting		
		WATERING	PRESSURE CON	WATER BOTTLE
1	Use the water supply	ON	0.15~0.2	OFF
2	Sanitation	OFF		ON
3	Use the bottle watering	ON		ON

- How to set the press control pressure

 Note	- The control pressure is set at 2 Bar (0.2) at the factory.
---	--

- Set as 0.15~0.2 by adjusting the volume with the gauge below (0.2 is recommended)



- Supplement the disinfectant

Image	Process
	1 . Put 60ml of 3% hydrogen peroxide solution and 940ml of distilled water in the sanitary solution container. 2 . Insert the sanitary solution bottle into the upper holder of the unit, and then turn it in the direction of the arrow to fix it.
	1 . When using sanitation, set it as shown in the image on the left.


 Warning	• The sanitary solution container may have a negative effect if washed at high temperatures, such as in a dishwasher or washing machine. Wash the water pipe with the water temperature below 40 °C. • If any damage such as cracks, discoloration, etc. is confirmed before filling the bottle, stop using it and replace the bottle with a new one.
--	--

 Note	• After cleaning the water pipe, remove the sanitary solution container and clean the water bottle with a neutral detergent and bottle brush.
---	---

- Sanitation Process

Button	How to use
	Press and hold the Water Drop button of Dr. Table membrane panel for more than 3 seconds to enter into the sanitation mode.

- Sanitation Process of Dr. Table

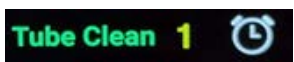Sequence	LCD Image	Process
1		1. When entering the water tube cleaning mode, the number 1 as shown on the left is displayed on the LCD is displayed and blinks.
2		1. Press the water drop button() on the Dr. Table membrane panel to inject disinfectant. 2. Repeat 3 cycles of total 1 minute each, 8 seconds for each 4 handpieces, 8 seconds for 3-way syringes, 10 seconds for cup, and 10 seconds for waiting.
3		1. Injection of the disinfectant is completed, and while waiting for the injection of the cleaning solution, The left and number 2 are displayed and flashing. 2. Standby state after completion of injection of disinfectant for cleaning solution injection 3. Turn off only the Water Bottle switch. 4. Turn off the power in standby mode. Go to work the next day and proceed to step (For efficient cleaning, it is recommended to wait at least 5 hours before cleaning in the water pipe line.)
4		1. During cleaning process, Change the setting of the distilled water + sanitation control module as shown on the left.
5		1. After turning off the power, turn on the next day, the left and number 2 are displayed on the LCD and blink. 2. When washing after disinfection without turning off the power, press the water drop button() on

		the Dr. Table membrane panel for more than 3 seconds to enter the water line washing mode again.
6		1. Press the water drop button of Dr. Table membrane panel to inject the cleaning solution. 2. 4 handpieces 8 seconds each, 3-way syringe 8 seconds, cup 10 seconds, standby 10 seconds, repeat 5 times in a 1 minute cycle.
7		1. When the disinfection solution injection is completed, the corresponding screen appears. 2. This shows the standby status after the cleaning solution injection has been completed.

 Note	• Water line flushing cannot be forced to end after disinfectant injection has started. Forced shutdown is possible only when waiting for the first disinfectant injection. • If the water drop button() is pressed or the handpiece is placed while disinfectant or cleaning solution is being injected, the operation is stopped and switched to standby mode, and the operation time is initialized. By pressing the button, you can proceed from the beginning again. • After water pipe washing is finished, put all instruments back to their original positions. • If the water bottle switch is not turned OFF after the disinfectant injection is finished, the injection continues until the disinfectant is exhausted in the bottle.
---	---

5. Maintenance after Use

- After use, turn off the system.
- Unplug the power cord when not in use for a long time.
- After use, remove foreign substances from the spittoon and wash it with water.
- To clean the inside of the drain or air suction pipe, suck it with clean water from the end.
- Periodically clean the spittoon filter and suction filter.
- Disinfect used instruments. If necessary, use a sterilizer. (Please follow each manufacturer's standards for disinfection method)
- Disinfect the handpiece after use and leave it in place (Please follow each manufacturer's standards for disinfection method)
- Clean the outside by spraying cleaning solution for preventing microbial infection or 80% ethyl alcohol on a clean cloth. Do not apply excessive force or use sharp objects.
- Do not allow liquid substances, water, or other foreign substances to penetrate into the inside of the device.
- When cleaning the surfaces of equipment and chairs, gently rub the surfaces with a soft, flat dry towel.



Warning

- Be sure to disconnect the power source before cleaning electronic device accessories.
- Do not fold cables made of electric wires (power, micro motor).
- Do not use compressed air for cleaning.

Equipment maintenance checklist

Division	Check list
Post-patient care review	- Disinfection and cleaning of contact surfaces after patient care - A wall where all splashes pop - Suction line check - Handpiece tube management - 3-way syringe and tip cleaning (using ultrasound) and sterilization - Handpiece cleaning, lubrication and sterilization - Wash the batting table for 1 minute

Every week	- Cleaning chair and stool covers with pure cleaning solutions and water - Suction valve cleaning (HVE and ejector) - Solids collector inspection and maintenance (twice a week) - Management of the oil collector - handpiece chuck oil ring - Clean the front of the dental light (use a soft cloth)
Monthly	- Oil collector maintenance - Replacement if worn and damaged after inspection of all handpiece and motor o-rings - HVE & ejector O-ring lubrication

6. Problem solving

Error phenomenon	First aid in use
Treatment chair does not move	1. Check that the power cable is connected correctly. 2. Check that the power switch is turned on. 3. Check the operation status of Dr. table + auxiliary table + foot switch. 4. Check if the foot controller pedal is depressed 5. Check if there is any pinching caused by objects under the treatment chair. 6. Confirm that the pinch between the armrest and the batting table has occurred
Handpiece does not work	1. Check that oil has been injected. (Once a day) 2. Check that the handpiece holder is properly positioned. 3. Check air supply condition
Low handpiece does not work	1. Check that the handpiece holder is correctly positioned. 2. Make sure the air pressure is correct. 3. Make sure the machine is turned on.
No water comes out of the handpiece	1. Check that the watering switch of the distilled water tank is set to City 2. Check that the water volume control valve at the bottom of Dr. table is open. 3. Check that the handpiece hole is not blocked. – Remove the handpiece and check that it works as a single tube.
Scaler does not work	1. Only water comes out but has no vibration 1.1 Check that the tip is not loosened 1.2 Check that the power control volume works correct 1.3 Make sure the cable is broken. 2. Vibration but no water 2.1 Check that the watering switch of the distilled water tank is turned on. 2.2 Check that the water volume control valve at the bottom of Dr. table is open 2.3 Remove the scaler body and check that it operates as a single item.
The water in the spittoon does not drain well	1. Check if the spittoon filter is clogged

No water comes out to the spittoon	1. Check the type of the cup 2. Check that the watering switch of the distilled water tank is set as City
Dental light not lit	1. Check that the power switch of the dental light head is turned on. 2. Check if the dental light is turned off by the non-contact power sensor

 Note	• Be sure to inspect the device in a state where there is no moisture or water • Be sure to turn off the power of the main unit before handling errors in the device. – The items that the user can take simple actions are as shown in the table above. If the problem persists, please contact nearest Dentium branch or local distributor.
---	---

7. Product Specification and Standard

7.1 Dental unit and chair

Item		Description
Product name		bright Chair 2
Model name		ICT-CH02S
Appearance (mm)		1843(W) x 1747(D) x 1685(H) Table: 623(W) x 497(D) x 332(H)
Weight		Approx. 340 kg
Rated power		200~240VAC, 50/60Hz, 1Ph
Power consumption		Max. 0.8 kVA
Protection type and degree		Class 1, Type B
Operating condition		Ambient temp. : 10 – 35°C Relative humidity : 20 – 75% Atmosphere : 800 – 1100hPa Others: There shall be no direct sunlight, flammable, or corrosive gas
Storage conditions		Ambient temp. : -5 – 45°C Relative humidity : 20 – 80% Atmosphere: 800 – 1100hPa Others: There shall be no direct sunlight, flammable, or corrosive gas
Water	regulations	Compliance with local drinking water standards
	water pressure limits	0.2~0.5MPa
	water hardness limit	5.8~8.5
	pH limits	300mg/L
	maximum particle size	40μm
	water flow rate limit	More than 5ℓ/min
Air	air pressure	0.5 ~ 0.7 MPa
	filter size	5μm
	flow rate	More than 5ℓ/min
Free warranty period		2 years

7.2 Electric Motor

Item	Description
Product name	bright elec
Model name	ICT-BE1
material	Titanium
Rotational speed and torque	Rotation speed: 3,000 – 40,000 rpm $\pm 10\%$ Rotation torque: 3.2Ncm $\pm 20\%$
Injection water supply	85ml/min $\pm 20\%$
Air supply volume	1.5NL/min
operating conditions	Action: 1 minute Stop: 10 minutes
weight and size	Weight: 70 grams Size: Ø 22 X 89(L)mm
tube length	1,400mm
usage environment	Temperature: 10 ~ 35°C Relative Humidity: 20 - 75% Atmospheric pressure: 800 - 1,100 hPa
Transportation and storage environment	Temperature: -5 – 45°C Relative Humidity: 20 - 80% Atmospheric pressure: 800 - 1,100 hPa
Free warranty period	1 year

7.3 Dental light

Item	Description
product name	bright light
Model name	ICT-BL1
type	LED (reflector method)
input voltage	DC 24V
power consumption	8VA (24V/0.3A)
beam size	165mm x 85mm ($\pm 20\%$)
color temperature	5,000k
color index (Color rendering index CRI)	at least 85 or more
Brightness	approx. 10,000 to 50,000 lux; preset 30,000 lux in accordance with ISO 9680:2015

usage environment	Temperature: 10 ~ 35°C Relative Humidity: 20 - 75% Atmospheric pressure: 800 - 1,100 hPa
Transportation and storage environment	Temperature: -5 - 45°C Relative Humidity: 20 - 80% Atmospheric pressure: 800 - 1,100 hPa
Free warranty period	1 year

7.4 bright Scaler

Item	Description
product name	bright scaler
Model name	ICT-BS1
type	Piezoelectric dental ultrasonic scaler
Output frequency	26 ~ 32Khz
Output adjustment	min 0.1~0.35A
weekly quantity	50mℓ/min ±20% (1bar)
operating conditions	Continuous operation
weight and size	Weight: 60 grams Size: Ø 18 X 153.5 (L)m
tube length	2,000mm
Input voltage and power consumption	DC 24V,
Water supply conditions	1 ~ 5bar
usage environment	Temperature: 10 ~ 35°C Relative Humidity: 20 - 75% Atmospheric pressure: 800 - 1,100 hPa
Transportation and storage environment	Temperature: -5 ~ 45°C Relative Humidity: 20 - 80% Atmospheric pressure: 800 - 1,100 hPa
Free warranty period	1 year

8. Labeling

8.1 Labels

Item	Description
Device name	Dental Chair, electric
Type name	ICT-CH02S
Manufacturing item license number	
Purpose of use	It refers to a medical treatment device and chair used for treatment in the dental field, and a treatment system is also configured.
Weight	340 kg
Performance and how to use	Refer to user manual
Precautions for use	Refer to user manual
Rated protection type and degree of protection	Class 1 device, type B device
Rated power	200~240VAC, 50/60Hz
Power Consumption	up to 0.8 kVA
Maximum allowable weight	135 kg

- ICT-CH02S Label information



bright Chair 2

Dentium

REF ICT-CH02S

LOT



Dentium Co.,Ltd. (ICT Branch)

(16229) 76 Changnyong-daero 256beon-gil,
Yeontong-gu, Suwon-si, Gyeonggi-do,
Republic of Korea
Tel +82-70-7098-8962 Fax +82-31-8019-9106
I083-01LA-02 rev.0

MADE IN KOREA Dental Chair, electric

Input Power : 200-240 V.a.c., 50/60Hz, 0.8kVA

Fuse Rated : 250V/6.3A, T,L

Protection Against Electric Shock : Class1

Duty Cycle (Chair) : 60s on, 600s off



MD

UDI

- ICT-01-FUNC Label information



Dentium
bright Chair 2
 Dentium Co.,Ltd. (ICT Branch)
(16229) 76 Changnyong-daero 256beon-gil,
Yeontong-gu, Suwon-si, Gyeonggi-do,
Republic of Korea
Tel +82-70-7098-8962
Fax +82-31-8019-9106
I083-02LA-01 rev.0

MADE IN KOREA
REF ICT-01-FUNC
Dental Foot Controller: FUNC
IP CODE : IPX1
Input Power:3.3VDC



- ICT-01-DISC Label information



Dentium
bright Chair 2
 Dentium Co.,Ltd. (ICT Branch)
(16229) 76 Changnyong-daero 256beon-gil,
Yeontong-gu, Suwon-si, Gyeonggi-do,
Republic of Korea
Tel +82-70-7098-8962
Fax +82-31-8019-9106
I083-02LA-02 rev.0

MADE IN KOREA
REF ICT-01-DISC
Dental Foot Controller: DISC
IP CODE : IPX1
Input Power:3.3VDC




9. Components

9.1 Basic components

Model	Basic components	Qty.	Note	Options
ICT-CH02S	Elec Low Motor	1		Seat Color, Air High Speed Handpiece, Air Low Speed Handpiece, Foot controller, Monitor Arm, Tissue Holder, Assistant Stool
	Scaler	1		
	3-way syringe	2		
	Saliva ejector	2		
	Vacuum syringe	1		
	Doctor's stool	1		
	Dental light	1		
 Note	We do not provide dental handpieces. It is recommended that users contact us before using any dental handpiece purchased. We are also not responsible for any risks/side effects associated with any dental handpiece you purchase.			

9.1.1 Tissue & Cup Holder

- These are chair-only accessories that are mounted on the chair to store tissues and cups for the convenience of patients during treatment.

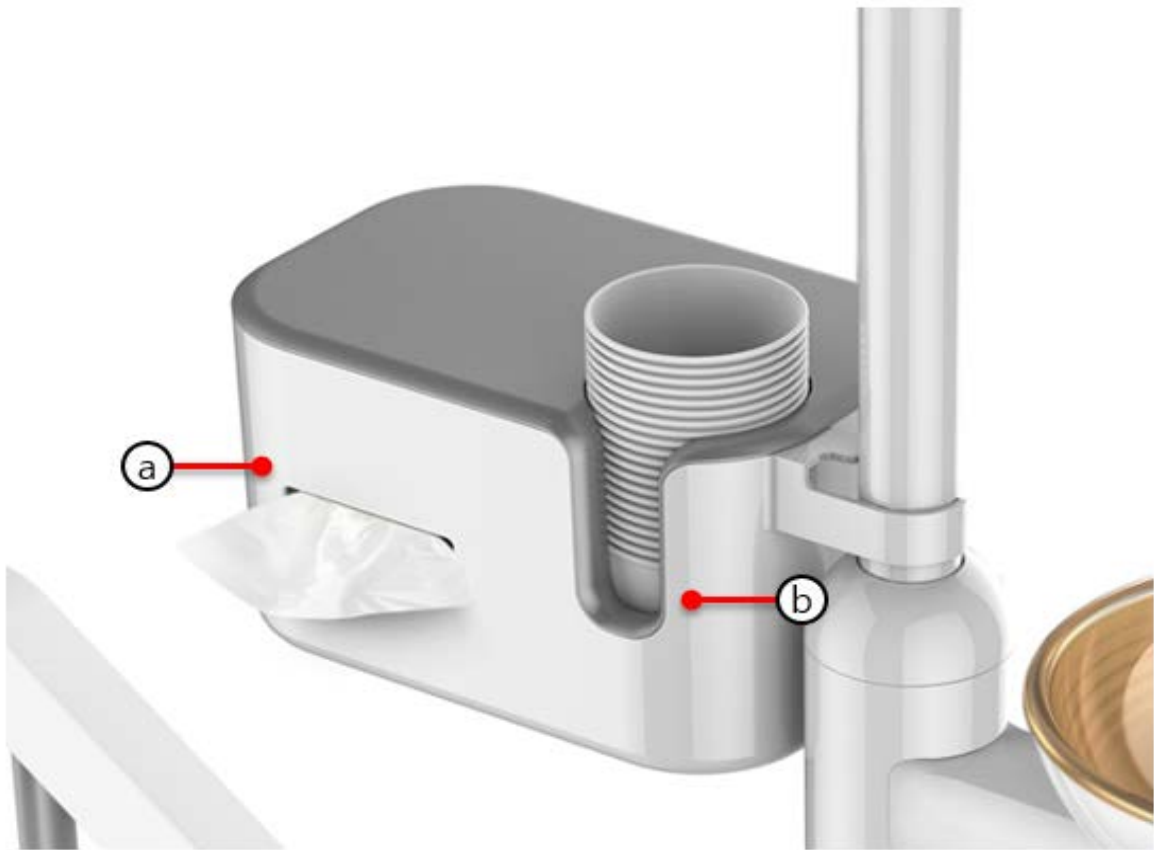

No.	Name	Function
Ⓐ	Tissue Holder	The part that holds tissues
Ⓑ	Cup Holder	The part that holds a cup



Warning


The holder may be damage if the load exceeding 1.5kg is applied to the holder.

9.2 Optional components

Option	Option components	Note
Seat Color	Indigo Navy	
	Ocean Blue	
	Dark Cyan	
	Brown	
Foot controller	Disk Type	
	Func Type	
Monitor Arm		
Tissue & Cup Holder		
Stool	Assist Stool	

9.2.1 Monitor Arm

- It supports the monitor and allows the user to fix the monitor in various positions desired by the user.

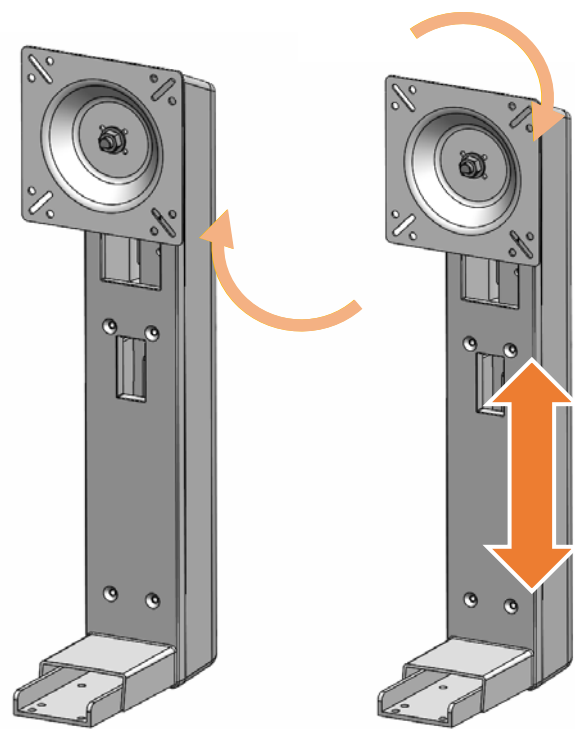



Note

- The applicable monitor size is from 17 inch to 24 inch.
- When you want to change the monitor or adjust the height of the monitor arm, please contact the nearest Dentium branch or local distributor.

9.3 Reference

High Speed Handpiece	S Max M900L	
	S Max Z900L	
	750TK-L	
	750TN-L	
Low Speed Handpiece	FX205, M65, M25L AD303, AD304	
Elec. Low Speed Handpiece	ICT-BE1	
Scaler	ICT-BS1	
	Varios 170	
 Note	We do not provide dental handpieces. It is recommended that users contact us before using any dental handpiece purchased. We are also not responsible for any risks/side effects associated with any dental handpiece you purchase.	

10 Appendix

10.1 Precautions for disposal of the system

- When disposing electric machines and accessories, they must be disposed of in accordance with the guidelines of the local laws and regulations.



- This mark means that electrical and electronic equipment waste cannot be disposed of as unsorted municipal waste must be collected separately.
- Before disposing of products or parts, contact the nearest Dentium sales office or local distributor to ensure that they are disposed of in a safe and appropriate manner.
- This equipment is basically designed to be disposed of without environmental pollution.

Details

Part name	Raw material	Whether to recycle or not	Hazardous material
Frame	Aluminum + Steel	◆	
Cover	Plastic	◆	
Sheet	Memory foam	◆	
Control board	Electronic components + PC	◆	
Cable	Copper + Steel + Oil	◆	
Packaging	Cardboard + foam	◆	

- This dental unit and chair Dentium ICT-CH02S is designed according to medical device regulations and manufactured and tested according to strict quality control system. Dentium Co., Ltd warrants that this warranty complies with the international warranty provisions of consumer protection laws.

10.2 Electronic Appliance Compatible Information

- This device is manufactured to operate in the electromagnetic environment described below.
- The customer or user must ensure that this device is being used in the environment described below.

Emission test	Standard	Electromagnetic Environment Guide
HF emissions according to CISPR 11	Group 1	The radiation properties of ICT-CH02S make it suitable for use in industrial areas and hospitals (CISPR 11 Class A). When used in a residential environment (where CISPR 11 Class B is normally required), this equipment may not provide adequate protection for radio frequency communication services. Users may be required to take mitigation measures, such as relocating or reorienting equipment.
Emissions according to CISPR 11	Class B Class A	ICT-CH02S is suitable for use in all establishments other than domestic use and can be used in homes directly connected to the public low-voltage power supply network. Warning This equipment should only be used by medical professionals. This equipment/system may cause radio interference and may interfere with the operation of nearby equipment. It may be necessary to take mitigation measures such as redirecting or relocating ICT-CH02S or shielding its location.
Harmonics according to IEC61000-3-2		
Voltage fluctuations/flicker according to IEC 61000-3-3	Compliance	

Radio interference resistance

- This device is designed to operate in the electromagnetic environment described below.
- The customer or user of this device must ensure that the device is being used in such an environment.

Radio interference resistance test	IEC60601-1-2-1-2 test level	Standard Compliance level	Electromagnetic Environment Guidelines
Electrostatic discharge (ESD) according to IEC 61000-4-2	8 kV direct contact discharge 15 kV air gap discharge	8 kV direct contact discharge 15 kV air gap discharge	Floors must be made with wood, cement or ceramic tiles. For floors made of synthetic materials, the relative humidity must be at least 30%

Radiated RF electromagnetic field immunity according to IEC 61000-4-3	3V/m 80MHz-2.7GHz 80% AM at 1kHz	3V/m 80MHz-2.7GHz 80% AM at 1kHz	ICT-CH02S is suitable for use in professional medical environments.
Transient fast electrical disturbance/burst according to IEC 61000-4-4	± 2kV power line ± 1kV I/O line 100 kHz repetition frequency	± 2kV power line ± 1kV I/O line 100 kHz repetition frequency	The quality of the supply voltage must be suitable for a typical business or hospital environment.
Surge voltage according to IEC 61000-4-5	±1 kV line to line ±2kV line and ground	±1 kV line to line ±2kV line and ground	The quality of the supply voltage must be suitable for a typical business or hospital environment.
Immunity to conducted disturbances induced by RF fields according to IEC 61000-4-6	3V 0.1580MHz 80% AM at 6V 1kHz in ISM band between 0.15~80MHz Power supply line & I/O line	3V 0.1580MHz 80% AM at 6V 1kHz in ISM band between 0.15~80MHz Power supply line & I/O line	The strength of the RF field in the frequency range higher than 150 kHz to 80 MHz, the strength of the RF field is less than 3V. The quality of the supply voltage must be suitable for a typical business or hospital environment.
Drops, interruptions and fluctuations in supply voltage according to IEC 61000-4-11	0 % UT: 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0 % UT; 1 cycle and 70 % UT; 25/30 Cycles Single phase: at 0°, 0 % UT; 250/300 cycle	0 % UT: 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0 % UT; 1 cycle And 70 % UT; 25/30 Cycles Single phase: at 0°, 0 % UT; 250/300 cycle	The quality of the supply voltage must be suitable for a typical business or hospital environment. It is recommended that the appliance have an uninterruptible power supply or battery to keep the appliance operating in the event of a power outage.
Supply frequency according to IEC 61000-4-8	3 A/m	3 A/m	If LCD display distortion occurs, it may be necessary to place the Model ICT-CH02S LCD display away from sources of power frequency magnetic fields or to install a magnetic shield. The power frequency magnetic field must be measured low enough at the intended installation location.
Note: UT is the mains AC voltage before using the test level.			

Electromagnetic Interference Resistance Test	IEC 60601-1-2 Test level	Standard Compliance level	Electromagnetic environment guidance
Induced HF disturbance /IEC 6100 0-4-6 Induced HF disturbance/ IEC 6100 0-4-3	3Veff 150 kHz ~ 80 MHz 3V/m 80MHz ~ 800MHz 3V/m 800MHz ~ 2,5GHz	3Veff 3Veff 3Veff	The use of portable and mobile wireless devices, including cables, should not be made closer to the device than the recommended protective distance. The recommended guard distance can be calculated using the appropriate equation for the transmit frequency. The recommended protection distance; $d = [1, 2] \sqrt{P}$ For 80 MHz to 800 MHz; $d = [2, 3] \sqrt{P}$ For 800MHz to 2.5GHz; P is the rated power (unit: W) of the transmitter according to the transmitter manufacturer's data, and d is the recommended protection distance (unit: m). The magnetic field strength of a stationary radio transmitter depends on the site survey, but must be less than the compliance level at all frequencies. Disturbance may occur in the environment of equipment marked with the following symbol. 

Protection distance

- Recommended protective distance between portable and mobile HF communication devices and this device
- This device is intended to operate in an electromagnetic environment in which radiated HF disturbances are controlled.
- RF communication equipment must be at least 30cm away from any part of ICT-CH02S, including cables designated by Dentium.
-
-
-

10.3 Installation and precautions

- Installation location



- The installation site must be a flat place, and it must be installed after securing enough free space.



Caution

This equipment must be installed by a qualified professional service engineer

- Select and install in a well-ventilated and level place.
- Avoid installation in places where there are dangerous substances such as explosives around.
- Securing equipment installation space: Secure 30cm distance from the wall [200*200*200cm].
- Product movement should be handled by an expert or by contacting the manufacturer.
- Ensure sufficient clearance (at least 20 cm to 30 cm) from the device and install it in a place where movement is controlled, such as on a wall or window.

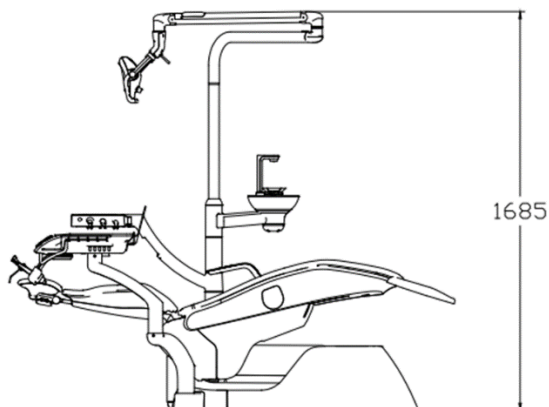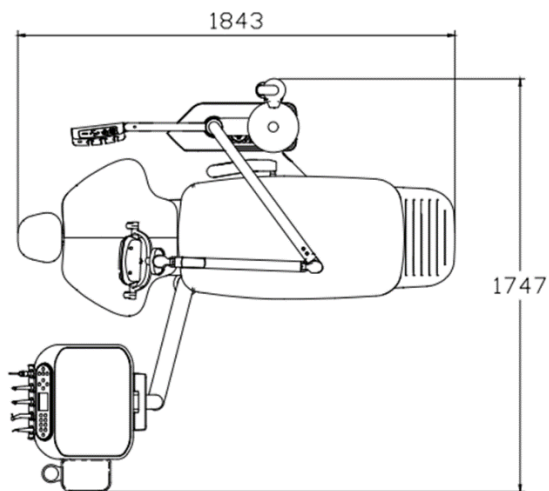

10.4 Manufacturer

Type name	Dental Chair, electric
Model name	ICT-CH02S
Manufacturer	Dentium Co., Ltd(ICT Branch)
Address	(16229) 76 Changnyong-daero 256beon-gil, Yeongtong-gu, Suwon-si, Gyeonggi-do, Republic of Korea
Tel/Fax	Tel +08-70-7098-8962 Fax +82-31-8019-9106

10.5 Warranty and Service

- Dentium Co., Ltd warrants to the purchaser that the ICT-CH02S and accessories are free from defects in material and workmanship under normal use and service for a specified period of two years from the date of purchase.
- The product warranty is conditional on the correct use of the product by the purchaser. During the applicable warranty period, Dentium Co., Ltd will repair or replace defective parts of the product free of charge to the buyer. However, if Dentium Co., Ltd determines that the product is not considered to be subject to warranty, the buyer will be responsible for paying all parts, shipping and labor costs required for repair.
- Upon purchase, please fill out the following warranty form and send it to the fax number below. (031-281-5561)
- When requesting the warranty, the customer shall present this warranty to the manufacturer.
- The service life of this device is 10 years.
- The following cases are not eligible for free warranty benefits.
 - 1) Defect or damage caused by accident, misuse, abnormal use, abnormal condition, improper storage due to the user's negligence.
 - 2) Damage or defects caused by external causes such as collision with objects, fire, flood, sand, earth, storm, lightning, earthquake, weather conditions.
 - 3) Defect or damage due to improper testing, operation, maintenance, installation, service, or adjustment not provided or approved by Dentium
 - 4) Defects or damage caused by ignoring warning signs in the user manual (instructions for use)

Device name	Dental Chair, electric	Manufacturing No.	
Model name			
Warranty period	From ~ to ~ (2 years) The warranty period of the products whose sales date is not recorded on the warranty is 2 years from the date of shipment from the factory.		

Product name	ICT-CH02S	Customer information	
Purchase date		Dental Clinic	
Manufacturing date		Address	
Serial No.			
Warranty period	2 years	Contact	

User manual Languages

BG	ES	CS	DA	DE	ET	EL	EN
							●
FR	GA	HR	IT	LV	LT	HU	MT
NL	PL	PT	RO	SK	SL	FI	KR

Printed in Korea

I083-01MU-02 rev.0

Dentium

(16229) 76 Changnyong-daero 256beon-gil, Yeontong-gu, Suwon-si, Gyeonggi-do,

Republic of Korea

Tel +82-70-7098-8962 Fax +82-31-8019-9106 WEB www.dentium.co.kr